
Instructions for Use

NEOPRENE WRAP AROUND
HINGED KNEE SUPPORT

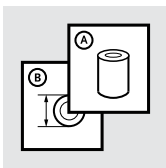3

EN | INSTRUCTIONS FOR USE

4

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5

EN – Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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ENGLISH

INDICATION FOR USE

Post-operation rehabilitation; tendonitis, bursitis, arthritis; and sprains and strains.

CAUTION

Patients should be instructed to inspect their knee and the product before and after each use. Modifications to this product are the sole responsibility of the prescribing physician.

PHOTO CAPTIONS

- a. Upper contact closure strap.
- b. Hinges.
- c. Anterior patella opening.
- d. Posterior opening.
- e. Lower contact closure.

FITTING INSTRUCTIONS

1. Position brace behind the knee so that the contact closures are open to the front of the patella. When opened, the wider section of the support is the top of the brace.
2. Secure upper contact closure, making sure that the closure is centered just above the patella.
3. Secure lower contact closure, making sure that the closure is centered just below the patella. Adjust straps as necessary for adequate support. Do not overtighten.
4. Confirm that patella is centered in the patella opening. Adjust if necessary.
5. Confirm the hinges are centered evenly around the knee. Adjust if necessary.
6. Secure the contact closure straps. Do not secure straps on/over the patella opening. Do not over-tighten. Adjustments may be necessary as knee swelling patterns change.

WASHING INSTRUCTIONS

1. The brace can be cleaned using a mild detergent and cold water (30°C / 86°F). Do not wash in washing machine.
2. Lay support out flat on a towel and air dry. Do not place in dryer. Do not place near heater.

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