## ENGLISH

***Indication for Use***

Conditions that may benefit from resting with the ankle in a neutral position, such as

- Plantar fasciitis
- Achilles tendinopathy

***Caution***

- Patient should be instructed to inspect their foot, ankle, and product before and after each use.
- Do not walk or attempt to place weight on the foot while using this splint without approval from your physician. This is not a weight-bearing product.
- Do not use outdoors, or on slippery or uneven surfaces.
- Do not use this product if plantar flexion contracture is greater than 10°.
- This product is not designed to treat uncontrollable, severe plantar flexion spasticity.

***Warning***

Do not over-tighten the brace, risking loss of blood flow. Patients with already compromised blood circulation in their extremities, like those with diabetes or peripheral vascular disease need to be extra careful and are advised to consult with their doctor before using the brace. If you experience pain, swelling, sensation changes, or if your extremity shows signs of insufficient blood flow (turns blue, white or cold) while using this brace, discontinue the use of it immediately and consult with your medical professional. Never wear the brace directly over an open wound.

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©Copyright Ossur 2018
**Ossur Deutschland GmbH**