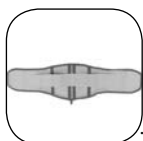
Instructions for use

Tri Back® Back Support

Life Without Limitations®

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	6
ⓕⓗ Guide de fabrication.....	8
ⓔⓈ Instrucciones para el uso.....	10
ⓔⓐ Istruzioni per l'uso.....	12
Ⓝⓞ Bruksanvisning.....	14
ⓓⓐ Brugsanvisning.....	15
Ⓢⓕ Bruksanvisning.....	16
ⓕⓙ Käyttöohjeet.....	17
Ⓝⓕ Gebruiksaanwijzing.....	19
ⓓⓐ Instruções de Utilização.....	21

EN - Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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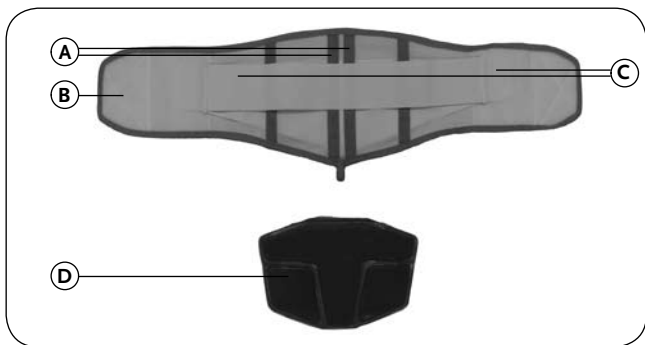
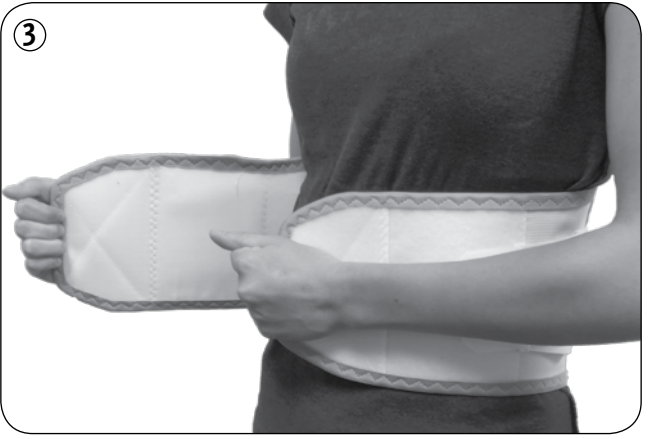
ENGLISH

USES

Muscular low back ache; acute strains; chronic low back pain; and lumbar support.

CAUTION

1. Patient should be instructed to inspect their back and this product before and after each use.
2. This product may be placed under or over clothing. DO NOT place over a belt.

PHOTO CAPTIONS

- A. Posterior Stays
- B. Contact Closures
- C. Double pull contact closure straps
- D. Hot/cold therapy gel pad (optional)

FITTING INSTRUCTIONS

1. If using hot/cold therapy, position gel pad on top of lumbar support.
2. Position the support so that posterior stays are centered in the middle of the back and buttocks.
3. Fasten the contact closures to secure the support into place.
4. For additional support, pull both double pull contact closure straps and adhere in front of the abdominal area.

HOT/COLD THERAPY

1. For cold therapy, remove gel pad from brace and place into a freezer for approximately 20 minutes to 1 hour. To reduce swelling, apply directly to the skin for 20 minutes.
2. For hot therapy, remove gel pad from brace and place in microwave oven for 20 second intervals at low setting until hot. Do not use oven or flame to heat the gel pad. Do not apply heated gel pad directly to skin. If too hot, allow to cool before applying.
3. Once gel pad is hot/cold, use contact closure tabs to adhere it into the brace to continue therapy. Diabetics and patients with vascular insufficiencies must always consult a physician before using hot/cold therapy.

WASHING INSTRUCTIONS

1. The brace may be cleaned using a mild detergent and cold water. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer. Do not place near heater. Do not use bleach or strong detergents.

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