

I N S T R U C T I O N S F O R U S E

Ossur North America
 27412 Aliso Viejo Parkway
 Aliso Viejo, CA 92656, USA
 Tel: +1 800 233 6263
 Tel: +1 949 362 3883
 Fax: +1 949 362 3888
 ossurusa@ossur.com

Ossur - Braces and Support
 742 Pancho Road
 Camarillo, CA 93012, USA
 Tel +1 805 484 2600
 Fax +1 800 284 4356

Ossur - Custom Braces
 19762 Pauling
 Foothill Ranch, CA 92610, USA
 Tel +1 949 859 4407
 Fax +1 800 453 4567

Ossur Spinal
 Mid Atlantic Corporate Center
 1414 Metropolitan Ave.
 P.O. Box 566
 Thorofare, New Jersey 08086, USA
 Tel: +1 856 848 0525
 Fax: +1 856 845 1148
 spine@ossur.com

Ossur Canada
 12111 Jacobson Way
 Richmond, BC
 V6W 1L5, Canada
 Tel: +1 604 241 8152
 Fax: +1 604 241 8153

Ossur Europe
 P.O. Box 120
 5690 AC Son en Breugel
 The Netherlands
 Tel: 00800 3539 3668
 Tel: +31 499 462840
 Fax: +31 499 462841
 saleseurope@ossur.com

Ossur Europe
 Kundenservice Deutschland
 Römerfeldstraße 2
 50259 Pulheim, Germany
 Tel: +49 (0) 2238 30 58 50
 Fax: +49 (0) 2238 30 58 01
 info-deutschland@ossur.com

Ossur Nordic
 Salagatan 16a
 P.O. Box 67
 751 03 Uppsala, Sweden
 Tel: +46 1818 2200
 Fax: +46 1818 2218
 info@ossur.com

Ossur International Markets
 Grjóthals 5
 110 Reykjavík, Iceland
 Tel: +354 515 1300
 Fax: +354 515 1370
 international@ossur.com

Ossur Asia Pacific
 2 Redbank Road
 Northmead NSW 2152
 Australia
 Tel +61-2-9630-9206
 Fax +61-2-9630-9268
 info-asiapacific@ossur.com



Sling & Swathe Shoulder Immobilizer

Part #	Size	U/M
13074	Universal	1 ea

Ossur Head Office
 Grjóthals 5
 110 Reykjavík, Iceland
 Tel + 354 515 1300
 Fax +354 515 1366
 mail@ossur.com



**SLING & SWATHE
 SHOULDER
 IMMOBILIZER**



Life Without Limitations

Sling & Swathe Shoulder Immobilizer



1



3



5



2



4



A.

B.

C.

USES

To immobilize the arm and shoulder after injury or post-op.

CAUTION

Patient should be instructed to inspect their arm, shoulder and this product before and after each use.

PHOTO CAPTIONS

- A. Swathe Wrap
- B. Contact Closure Strap
- C. Sling

FITTING INSTRUCTIONS

1. Position sling against rib cage. Place affected arm through the sling.
2. Bring contact closure straps straight up and over the patient's shoulders.
3. Criss cross both straps against the patient's back.
4. Attach hook straps to the front of the sling in a criss cross motion supporting the affected arm. Trim excess strap length.
5. Secure arm against the body by attaching the swathe strap to the sling. Place one end of the strap to the sling. Encircle the affected arm and bring around the patient's back. Secure loose end to the front of the sling.

WASHING INSTRUCTIONS

1. This product may be cleaned using a mild detergent and cold water. Do not use bleach or strong detergents. Do not wash in washing machine
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater.