ENGLISH

USES

Clavicle fractures and dislocations, selected soft tissue injuries and postural problems.

FITTING INSTRUCTIONS

1. Position the felt pad on back between the shoulder blades.
2. Wrap figure-8 arm straps around shoulders. Make certain the straps are not twisted and the arm pads are between the straps and the shoulders.
3. Feed the straps through the buckles and pull tight as desired.
4. The arm pad should be adjusted so that it fits under the arm and in front of the shoulder. The straps should be tight enough to pull the shoulders back.

WASHING INSTRUCTIONS

This product may be cleaned using mild detergent and cold water. Do not bleach. Do not wash in washing machine or place in dryer. Lay brace flat on a towel to air dry. Do not squeeze.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©Copyright Össur 2019