<ul style="list-style-type: none">Material: Aluminium. Weight Limit: 45kg. Weight: 50g.		
	Torque	Loctite
Mounting Screws	6 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.

May be used for Transtibial and Transfemoral laminated sockets.

INSTRUCTIONS FOR INTEGRATION INTO LAMINATED SOCKETS.

Socket must be laminated in two layers. The 4-Hole Socket Adapter should be attached to the inner layer with putty. All surfaces should be roughened prior to second lamination.

- Isolate cast with PVA bag.
- Apply appropriate lay up material for first lamination.
- Apply outer PVA bag and laminate.
- Once cured, remove PVA bag and roughen outer surface of lamination.
- Attach 4-Hole socket adapter in correct alignment and rotation with putty.
- Position Lamination Cap on Adapter and secure using a 3mm Hex key.
- Seal putty and Lamination Cap with sealing resin.
- Apply Lay up material and combinations of material appropriate to the individuals weight and impact/ activity level.

IMPORTANT: Tie 50mm carbon reinforcements into both grooves, between lamination cap and adapter, and groove in adapter

The Lamination Cap should be removed following lamination by unscrewing the bolts using a 3mm Hex key.

- 2 year limited warranty.**

Aluminum tested to ISO10328, load level A60, rated to 100lbs/45kg, for normal use.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Ossur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Ossur products are used with other recommended Ossur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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