<ul style="list-style-type: none">Material: Titanium Weight limit: 166kg Height: 60mm Weight: 208g		

	Torque	Loctite
Set Screw	15 Nm	243
Slide Adjustment Screw	12 Nm	243
Slide Limit Screw	5 Nm	

The Shift Adapter provides adjustment up to 11mm in both the frontal and sagittal planes according to the adapter orientation, in order to achieve optimal prosthetic alignment.

Security provided from the Limiting screw restricts shift movement in one direction. It is important that the Limiting Srcew may be removed and located into the second hole to provide shift movement in the opposite direction.

When sliding in frontal plane the slide adjustment screws must be positioned anteriorly.

Torque slide adjustment screws until both have reached desired torque.

Ensure the Shift Adapter is positioned with the female shift section (A) proximally and the male shift section (B) distally on the prosthesis. This orientation provides maximum strength from the adapter.

- 2 year limited warranty.**

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2017 IFU 0158 1171_001 Rev. 5
Össur Deutschland GmbH