# Instructions for use

Exoform<sup>®</sup> Wrist

*Life Without Limitations<sup>®</sup>*

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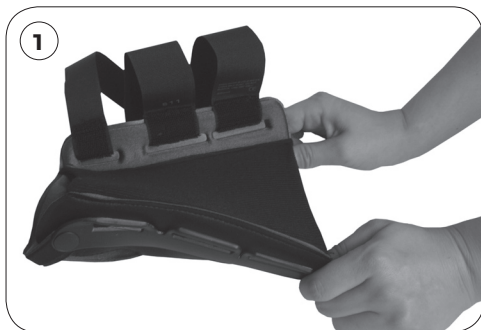
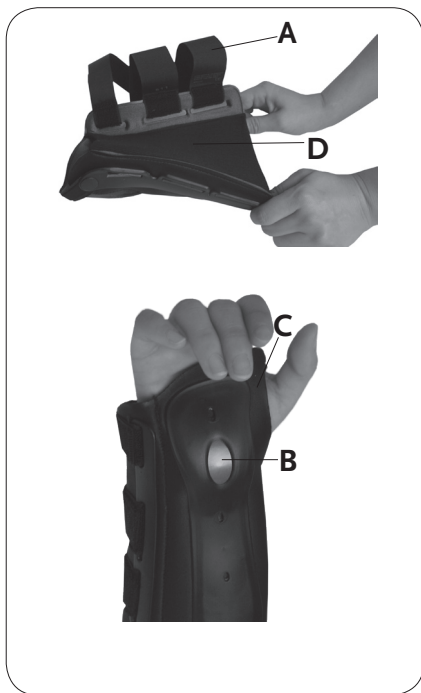
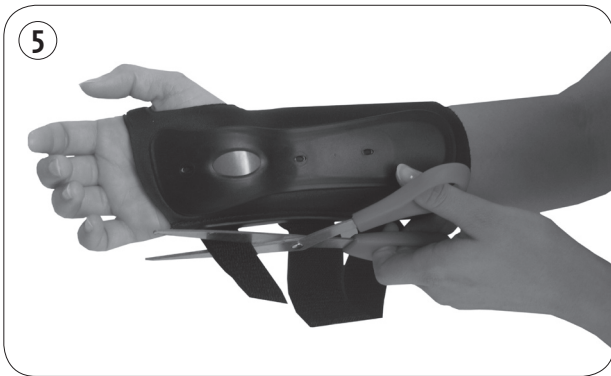
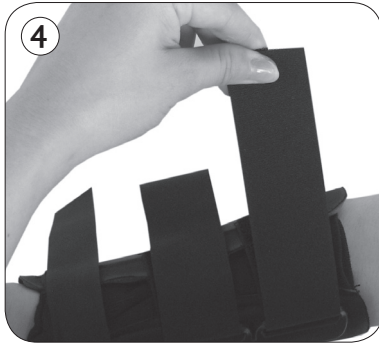
Exoform® Wrist is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.

EP: 690703; 1265565.

USA: 5,695,452; 7,311,686.

## USES

Tendonitis, Carpal Tunnel Syndrome, arthritis, strains and sprains, post cast healing and other selected injuries

## CAUTION

Patients should be instructed to inspect their hand, wrist, and product before and after each use.

## PHOTO CAPTIONS

- A. Adjustable contact closure straps
- B. Adjustable palmer stays
- C. Thumb hole
- D. Stockinet

## FITTING INSTRUCTIONS

1. Undo all the contact closures.
2. Place hand into the support and place thumb into thumb hole. Ensure stockinette is positioned on top of the hand
3. Align the palmer stay to the palm of the hand. Adjustments on the aluminum palmer stay can be performed by a certified medical professional.
4. Fasten all contact closure straps and secure to a snug fit. Adjustments may be necessary as hand, wrist, and thumb swelling patterns change. Straps may be cut as desired.

## WASHING INSTRUCTIONS

1. This product may be cleaned using a mild detergent and cold water. Aluminum stays may be removed for cleaning except for the Exoform Wrist Brace. Do not use bleach or strong detergent. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not wash in washing machine or place in dryer. Do not place near heater.

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