# Instructions for use

OAsys<sup>®</sup> Carticare

*Life Without Limitations<sup>®</sup>*

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	7
ⓕⓗ Guide de fabrication.....	9
ⓔⓈ Instrucciones para el uso.....	11
ⓔⓐ Istruzioni per l'uso.....	13
Ⓝⓐ Bruksanvisning.....	15
ⓓⓐ Brugsanvisning.....	17
Ⓢⓕ Bruksanvisning.....	19
ⓕⓓ Käyttöohjeet.....	21
Ⓝⓓ Gebruiksaanwijzing.....	23
ⓓⓐ Instruções de Utilização.....	25

**EN - Caution:** This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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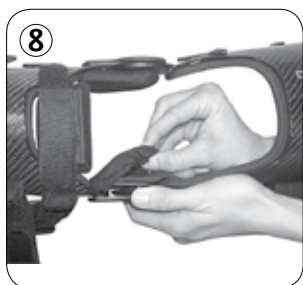
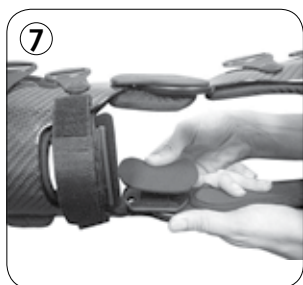
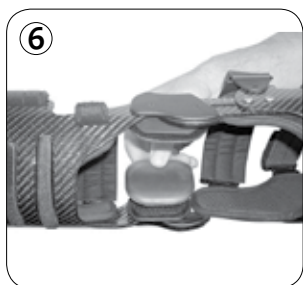
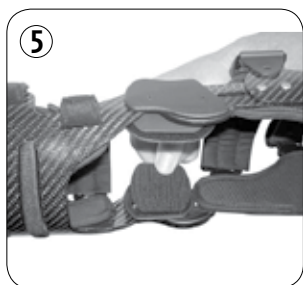
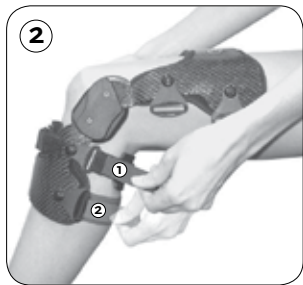
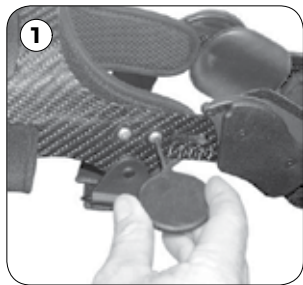
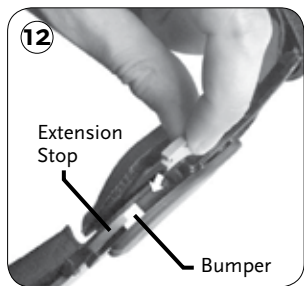
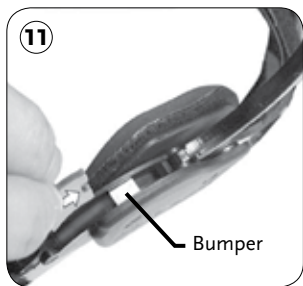
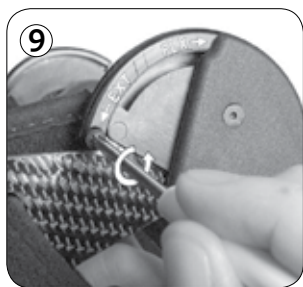
- I. 1/4" Molded Condyle Pads (2)
- J. Hex Wrench

## APPLICATION

**Initial application and adjustment of the OAsys CartiCare knee brace, as well as all subsequent adjustments to the Acculign™ Slide Bar, should be performed by a healthcare provider.**

1. Loosen the hex screws securing the Acculign Slide Bar (**Figure 1**).
2. Unfasten all quick release buckles.
3. With leg bent at 45°, place brace over knee, with lateral arms resting on outside (buckle side) of leg, so bottom of condylar pads are aligned with base of knee cap.
4. Flex and extend leg until slide bar adjusts to the natural alignment of the leg.
5. Beginning with the strap immediately below the knee, loosen the Velcro™ tab closest to the quick release buckle, fasten the buckle and re-tighten the strap. Be sure the strap is secured above the calf muscle to aid in the suspension of the brace. Repeat the above steps for the bottom strap (**Figure 2**).
6. Repeat the same procedure for the strap immediately above the knee, followed by the top strap (**Figure 3**).
7. Adjust the Acculign Slide Bar to achieve the desired amount of unloading (**Figure 4**). The slide bar has a lettered scale and may be moved in either direction to increase or decrease the amount of varus or valgus force on the leg. The lettered scale does not relate to degrees of unloading. Rather, each letter corresponds to approximately 2 1/2°. All adjustments should be determined by a healthcare provider, in accordance with the patient's comfort level. Each time the brace is adjusted, the healthcare provider should note the letter for a record of progression.
8. Tighten hex screws to secure Acculign Slide Bar.

**NOTE: #1, 4, 7 and 8 only apply when making adjustments to the Acculign slide bar.**

## PADDING ADJUSTMENTS

**To narrow the fit at the knee joint:**

Velcro™ shims (provided) can be installed as follows:

1. Remove the gel pad.
2. Attach shim to the Velcro™ fastener on which the gel pad was previously seated (**Figure 5**).
3. Replace the gel pad on top of the shim (**Figure 6**).
4. Repeat for other side, if necessary.

**To widen the fit at the knee joint:**

The installed gel pads can be removed and replaced with thinner pads (provided). Three additional sets of padding are included with your OAsys CartiCare brace: 1/4" molded, 1/8" Neoprene and 1/16" Neoprene. Any combination of pads can be used to achieve a more comfortable fit and alleviate pressure at the joint. When using the molded pads, the larger of the two pads should be placed on the opposite side of the affected area.

1. Remove gel pads and replace them with the desired pads (**Figure 7**).
2. If desired, cover pads with provided covers (**Figure 8**).

## FLEXION/EXTENSION STOPS

The OAsys CartiCare™ knee brace provides range of motion (ROM) control, from full ROM to "lockout" (i.e., no flexion allowed). The amount of flexion and extension can be adjusted by changing the size of the flexion and extension stops. Changing stops should be done under the advice of your healthcare provider. If you change the stop on one hinge, you **MUST** change the stop to the same size on the opposite hinge.

**EXTENSION STOP SIZES:** 10°, 20°, 30°, 40°, Lockout (full ext)

**FLEXION STOP SIZES:** 45°, 60°, 75°, 90°

**To adjust flexion and extension stops:**

1. Remove screw securing stop cover. NOTE **EXT** and **FLX** markings on side of stop cover indicating placement of extension and flexion stops (**Figure 9**).
2. Lift stop cover and insert desired extension stop into top half of groove (**Figure 10**).
3. Make sure notch in extension stop faces towards metal bumper located in middle of groove (**Figure 11**).
4. Insert desired flexion stop into lower half of groove with notch facing towards metal bumper (**Figure 12**).
5. Close stop cover and replace screw.

**USE & CARE**

The OAsys CartiCare knee brace has been designed to last. However, as with any product, its longevity will be affected by how frequently the brace is used, and by the quality of care the brace receives. By following the simple guidelines below, you can insure that you will get maximum usage from your OAsys CartiCare brace:

- Your straps, condyle pads and condyle covers are washable and should be cleaned periodically with a mild detergent. Hand wash and line dry for best results. Automatic washer and dryer not recommended.
- If you will be using the brace in salt water, be sure to rinse the brace and the straps with fresh water after each use to remove any salt or debris that might remain.
- Always use an oversleeve when participating in water sports to insure that straps will remain firmly attached in high speed falls.
- Keep brace frame clean by periodically washing it with a mild detergent and rinsing with clear water.
- Keep the hinge mechanism clean by rinsing away dirt, sand or debris with fresh water. Lubricate hinges with a silicone or dry graphite spray lubricant to keep them moving freely and quietly. **DO NOT** use oil based lubricants (i.e., WD40™), as the oil attracts dirt to the hinge mechanism.

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OAsys® Carticare is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.