Instructions for use

AirForm® Night Splint

Life Without Limitations®

EN - Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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Plantar Fasciitis, Achilles tendonitis, drop foot and post-static pain

CAUTION

1. The patient should be instructed to inspect their foot, ankle, and product before and after each use.
2. Do not walk or attempt to place weight on the foot while using this splint without approval from your physician. This is not a weight-bearing product.
3. Do not use outdoors, or on slippery or uneven surfaces.
4. Contact your certified medical professional if any problems occur with the use of this product.
5. Do not use this product if plantar flexion contracture is greater than 10°.
6. This product is not designed to treat uncontrollable, severe plantar flexion spasticity.
7. Patients with foot neuropathy (insensitive feet) should not use this product.

PHOTO CAPTIONS

- A. Calf strap
- B. Brace
- C. Toe strap

FITTING INSTRUCTIONS

1. Undo toe and calf straps.
2. Step foot into brace.
3. Secure toestraps by affixing the contact closures.
4. Apply calf strap by wrapping behind the leg and affix contact closures.

WASHING INSTRUCTIONS

1. This product may be cleaned at any time using a mild detergent and cold water. Do not use bleach or strong detergents. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater.

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Össur Iberia S.A.