## ENGLISH

### CONTENT

- Housing
- O-ring
- Retainer
- Valve
- Membrane
- Dummy for Rigid Sockets (RED)
- Dummy for Flexible Sockets (WHITE or GREEN)
- 2x Nails

### TOOLS REQUIRED

- 2mm (1/16") Drill bits
- Hammer

### CAST RECTIFICATION

- Determine the valve location on the plaster model and flatten the area with a Sureform to ensure the dummy sits flush. In the center hole of the dummy place a mark on the plaster model with an indelible marker.
- Drill a 2mm (1/16") vacuum hole using a long drill.
- Attach the dummy to the plaster model, using the nails provided.

### IMPORTANT:

For rigid sockets choose RED dummy.

For flexible inner sockets choose WHITE or GREEN dummy.

#### PREPARATION FOR THERMOPLASTIC SOCKETS

- Drape the thermoplastic socket.
- Guide the plastic around dummy to ensure proper molding.

#### PREPARATION FOR LAMINATED SOCKETS

- Place the first PVA bag over the plaster model with dummy in place.
- Place the reinforcing layers and second PVA bag over the dummy. After mixing the resin apply the vacuum to remove trapped air. Pour the resin.

#### FINISHING AND INSTALLATION

- Expose dummy by grinding away material until fully flat area is exposed.
- Remove dummy and insert housing with O-ring.
- Firmly hand tighten the retainer to the housing. Make sure the O-ring is not visible from inside the socket.

#### DISASSEMBLY AND CLEANING

- Unscrew valve and remove the membrane.
- Clean under running water with mild detergent and clear airways with an air hose. Hold down the button in the cap while clearing airway.

#### REASSEMBLY

- Replace the membrane with convex surface facing down.

#### PRODUCT WARRANTY

- OSSUR offers 6 months limited warranty on manufacturing faults.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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© Copyright Össur 2017
**Össur Deutschland GmbH**