## ENGLISH

### USES

Mild to moderate plantar ulcers and lesions

### CAUTION

- Hot and/or cold therapy should not be used with this product.
- This product has been designed for the treatment of ulcers that are located on the plantar aspect of the foot. This product should not be used on patients with bony deformities.
- This product has been designed and tested based on single patient usage and is not recommended for multiple patient usage.
- The use of this product is left to the discretion of the prescribing physician.
- Modifications to this product are the sole responsibility of the prescribing physician. Hexes should be only removed and altered when instructed by the prescribing physician.
- The Active Offloading™ insole should not be used in conjunction with any other footwear device.
- If the relieved zone is too large and inhibits offloading and pressure relief by allowing the ulcer to bottom out, this product should not be used.
- In order to prevent individual hexes from collapsing into the relieved area, trim (scythe) the corners of the hexes lying on the perimeter of the relieved area. This method should only be performed when warranted by the prescribing physician and should only be performed by the prescribing physician.
- It is the prescribing physician's responsibility to monitor the patient's progress. Be sure the patient performs circulation checks. If a loss of circulation is felt, or if the shoe feels too tight, release contact closure straps and adjust to a comfortable level. If discomfort continues, the patient should contact the prescribing physician immediately.

### PHOTO CAPTIONS

- Removable Toe Piece
- Contact Closure Flap
- Outsole
- Hexagonal Insole
- Heel Counter
- Ankle Strap Pad
- Contact Closure Ankle Strap
- Toe Piece Tongue

### FITTING INSTRUCTIONS

- Remove hexagonal insole and toe piece from the shoe. Remove toe piece from insole.

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## ENGLISH

2. Re-insert insole into shoe. Make sure the insole is centered evenly and there is no gap between the insole and the heel counter. The insole must be positioned correctly for accurate marking.

- Cover the ulcer with tape or gauze. Mark the tape or gauze with a marking material. Examples of marking materials include iodine and lipstick.
- Place foot inside the shoe, heel first. Fasten ankle straps and flaps. To mark the hexes, have the patient stand. Heel should be positioned comfortably against the heel counter.
- Remove foot from shoe and remove marked hexes. Scythe hex edges around the relief zone with a sharp cutting tool. Remove tape or gauze from foot. Scything hex edges like as shown in picture 5 to prevent the hexes from falling into the relief zone.
- For open-toe option: Place foot inside the shoe, heel first. Position ankle strap pad such that it is centered over foot. Fasten the ankle strap and flaps. Ankle strap pad slides along the length of the ankle strap. The strap needs to be centered correctly to protect the foot from the ankle strap. Trim excess strap length.
- For closed-toe option (Steps 7 through 8): Lift front section of insole from outsole of shoe. Insert toe piece onto front of insole. Make sure there is no gap between front of insole and inside of toe piece. Do not remove entire insole from the shoe to keep the orientation of the offloading.
- Insert foot into shoe, toe first. Fasten ankle straps and flaps. Center the ankle strap pad for extra cushioning. Tongue should be positioned under the ankle strap in order to provide cushioning.

### WASHING INSTRUCTIONS

- Wash shoe and insole separately.
- Remove insole from shoe. Rinse the insole with gently flowing 86F (30° C) warm water.
  - Wash shoe by hand using mild soap, a damp cloth or sponge, and 86F (30° C) warm water. Do not remove any additional hexes. Do not use alcohol or solvents. Do not put in washing machine or immerse shoe in water.
  - Insole should be air dried. To speed drying, a hair dryer can be used on the low setting.
  - When the pad is fully dried, re-insert into the shoe. Always follow fitting instructions when re-inserting the insole into shoe.

### PHOTO CAPTIONS

- Removable Toe Piece
- Contact Closure Flap
- Outsole
- Hexagonal Insole
- Heel Counter
- Ankle Strap Pad
- Contact Closure Ankle Strap
- Toe Piece Tongue

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**EN – Caution:** This product has been designed and tested as a single use product, and is not intended for multiple use. In addition to increasing the risk of infection, used product components may contain small unnoticeable defects that may lead to non-optimal performance, and even mechanical failure. If any problems occur with the use of this product, immediately contact your medical professional.

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Össur UK Ltd