ENGLISH

INDICATION FOR USE

Ideal for conditions that may benefit from increased coronal plane control, such as:

- Patellofemoral Pain Syndrome (PFPS)
- Patellofemoral Stress Syndrome (Runner's Knee)
- Patellar Tendinopathy (Tendonosis or Jumper's Knee)
- Patellar Subluxation/Dislocation
- Chondromalacia
- Arthritis
- Osgood-Schlatter Disease

CAUTION

Always consult a health care professional before starting any course of rehabilitation.

CAUTION: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions

PHOTO CAPTIONS

- A. Patella Opening
- B. Side Stays
- C. Thigh Strap
- D. Coronal Strap
- E. Buttress
- F. Coronal Adjuster

FITTING INSTRUCTIONS

1. Unfasten thigh strap, coronal strap, and coronal adjuster. Place foot into brace and slide on leg so that patella opening is centered over the patella.
2. Fasten coronal strap using hook and loop until comfortable, do not overtighten.
3. Fasten coronal adjuster onto coronal strap by pulling down as shown.
4. Fasten thigh strap.

NOTE: Buttress can be removed and adjusted if necessary.

WASHING INSTRUCTIONS

1. Hand wash in cold water. Do not use chlorine. Hang to dry.
2. Wash well if sweat pattern forms on outside of brace.

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