Instructions for use

Innovator DLX<sup>®</sup>

*Life Without Limitations<sup>®</sup>*

**EN - Caution:** This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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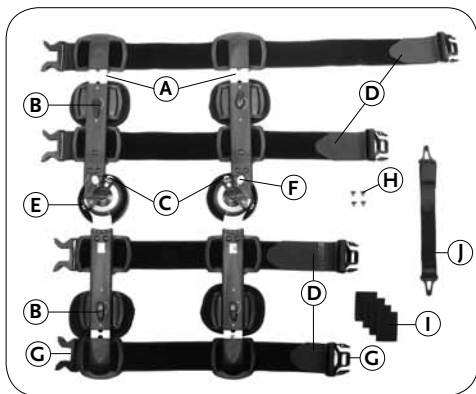
---

## USES

Controlled range-of-motion for post traumatic, post-surgical repair of the knee.

## CAUTION

1. This product has been designed and tested for single patient use and is not recommended for multiple patient usage.
2. If the patient experiences increased pain, swelling, sensation changes or any adverse reactions while using this product consult your medical professional immediately.
3. Always consult your physician or therapist before making any adjustments to the brace.
4. Proper rehabilitation and activity modification are also essential parts of a safe treatment program.
5. Federal law prohibits this device to be sold or ordered by anyone who is not a licensed health care practitioner.

## PHOTO CAPTIONS

- A. Adjustable/Removable Strut
- B. Length Adjustment Lever
- C. Flexion Setting
- D. Adjustable Straps (Premium 6: Cool 4)
- E. Extension Setting
- F. OneTouch™ Drop-Lock Button
- G. Buckles (Premium 6; Cool 4)
- H. ROM locks
- I. Double Sided Hook
- J. Gastroc/Calf Strap

## FITTING INSTRUCTIONS

1. Secure and fasten all softgoods. Ensure the patella remains exposed and that the medial and lateral hinges are aligned just above mid-patella.
2. To determine brace length, position the hinge pivot center slightly above the knee joint midline.
3. To increase length of the strut, rotate length adjustment lever one-half turn (180 degrees) to the unlocked position.
4. Position and alignment marks are provided on the metal strut. When desired side strut length has been achieved, rotate length adjustment lever back to the locked position. Strut is preset at 23" (58 cm). Do this adjustment for both hinges. To decrease length of strut, the extension bars may be completely removed to minimize length to 17" (43 cm). If extension bars are removed on cool version brace replace the pad on the adjacent strut D-ring with cool softgood and strap.
5. After length is determined, trim softgood padding as necessary. Cool softgoods should not be trimmed.
6. Fasten buckles and adjust strap length, starting with the two straps closest to the knee. Straps may be trimmed if necessary. If straps are trimmed, secure straps using supplied hooks (I).
7. Apply anti-migration (gastroc/calf) strap (J) as required (optional).
8. To adjust range of motion, press the flexion or extension button and rotate until the desired degree of flexion or extension appears through the indicator window.
9. To set the hinge lock, determine the desired lock angle. Align the setting arrow on the hinge to the appropriate angle as marked on the lower side bar sleeve. Slide the drop lock button to secure the angle of the hinge.
10. To enforce compliance, insert supplied ROM locks (H) into the holes in the flexion and extension buttons to limit range of motion adjustment.
11. The side bars may be bent to fit individual anatomy. Side bars must be bent with final length adjustments completed and with metal struts installed in the plastic sleeves. Bend with bending irons.
12. To remove the brace, release buckles and open the foam pads. The brace can then be removed and reapplied as a single unit.

## WASHING INSTRUCTIONS

1. This product may be cleaned using a mild detergent and cold water. Remove hinge assembly before washing. Do not use bleach or strong detergents. Do not wash in washing machine or place in dryer.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater.

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**Össur Iberia S.A.**