NORSK

ANVENDELSE

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EN - Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

©Copyright OSSUR 2009
www.ossur.com
mail@ossur.com
Tel: +354 515 1300
Tel: +354 515 1300
Reykjavik Iceland
Ortohals 5
OSSUR Head Office

info-europe@ossur.com
Fax: +31 499 46241
Tel: +31 499 46240
传真: +86 21 6127 1799
电话: +86 21 6127 1700
The Netherlands
Tel: +800 339 3668
5690 AC Son en Breugel
P.O. Box 120
Ekkersrijf 4106-4114
OSSUR Europe
Tel: +44 161 490 8501
Fax: +44 161 490 8500
Manchester M22 5TC, UK
Avatar Way
Manchester Business Park
Billing 3000
OSSUR UK
Tel: +1 604 241 8153
Fax: +1 604 241 8152
VIA OLI, Canada
Richmond, BC
120-11231 Dyke Road
OSSUR Canada
Tel: +1 949 362 3888
Fax: +1 949 362 3888
Tel: +800 233 6263
2832 Las Rozas - Madrid
Calle Edimburgo N°14
Parque Európolis
Aliso Viejo, CA 92656, USA
2712 Aliso Viejo Pkwy
OSSUR Americas
ossurusa@ossur.com

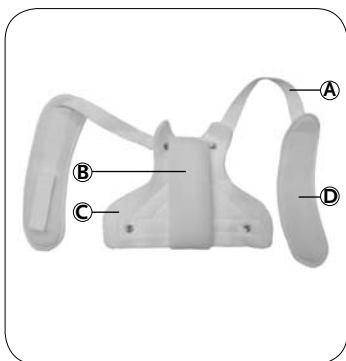
Instructions for Use

Front Closure Clavicle Support

Life Without Limitations™

ENGLISH

USES

Clavicle fractures and dislocations, selected soft tissue injuries and postural problems.

CAUTION

Patients should be instructed to inspect their back, shoulders and this product before and after each use.

PHOTO CAPTIONS

- A. Arm straps
- B. Removable foam insert
- C. Felt pad
- D. Arm pad

FITTING INSTRUCTIONS

1. Position the removable foam insert vertically on the felt pad.
2. Place arms through arm straps so that felt pad is located on back and the foam insert is between shoulder blades.
3. Adjust by pulling the arm straps tight. Fasten the contact closures on the arm pad.
4. The cushioned arm pad should be adjusted so that it fits under the arm and in front of the shoulder. The straps should be tight enough to pull the shoulders back.

WASHING INSTRUCTIONS

This product may be cleaned using mild detergent and cold water. Remove foam insert before washing. Do not bleach. Do not wash in washing machine or place in dryer. Lay brace flat on a towel to air dry. Do not squeeze or wring out brace.

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