## DANSK

**ANVENDELSE**

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## ENGLISH

### USES

Muscular low back ache; acute strains; chronic low back pain; and lumbar support

### CAUTION

1. Patient should be instructed to inspect their back and this product before and after each use.
2. This product may be placed under or over clothing. DO NOT place over a belt.
3. Be certain the pump nozzle is completely inserted into the valve. Too much air can be uncomfortable. Adjust as necessary.

### PHOTO CAPTIONS

- A. Air intake system
- B. Adjustable dual air bladder lumbar support
- C. Double pull contact closure strap
- D. Posterior stays
- E. Hot/cold therapy gel pad(optional)

### FITTING INSTRUCTIONS

1. If using hot/cold therapy, position gel pad on top of lumbar support.
2. Position the support so that posterior stays(2) are centered in the middle of the back and buttocks.
3. Fasten the contact closures to secure the support into place.
4. For additional support, pull the double pull contact closure straps from the left side and adhere in front of the abdominal area. Repeat on the right side.
5. To inflate: fully insert clear tip of bulb into valve. The valve on the right inflates right air cell. The valve on the left inflates left air cell.
6. To deflate: fully insert black tip on the end of the bulb into valve.

### HOT/COLD THERAPY

1. For cold therapy, remove gel pad from brace and place into a freezer for approximately 20 minutes to 1 hour. To reduce swelling, apply directly to the skin for 20 minutes.
2. For hot therapy, remove gel pad from brace and place in microwave oven for 20 second intervals at low setting until hot. Do not use oven or flame to heat the gel pad. Do not apply heated gel pad directly to skin. If too hot, allow to cool before applying.
3. Once gel pad is hot/cold, use contact closure tabs to adhere it into the brace to continue therapy. Diabetics and patients with vascular insufficiencies must always consult a physician before using hot/cold therapy.

### WASHING INSTRUCTIONS

1. The brace may be cleaned using a mild detergent and cold water. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer. Do not place near heater. Do not use bleach or strong detergents.

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