

Össur Americas

27051 Towne Centre Drive
Foothill Ranch, CA 92610, USA
Tel: +1 (949) 382 3883
Tel: +1 800 233 6263
ossurusa@ossur.com

Össur UK Ltd

Unit No 1, S:Park
Hamilton Road
Stockport SK1 2AE, UK
Tel: +44 (0) 8450 065 065
ossuruk@ossur.com

Össur Europe BV

De Schakel 70
5651 GH Eindhoven
The Netherlands
Tel: +800 3539 3668
Tel: +31 499 462840
info-europe@ossur.com

Össur Deutschland GmbH

Augustinusstrasse 11A
50226 Frechen
Deutschland
Tel: +49 (0) 2234 6039 102
info-deutschland@ossur.com

Össur Canada

2150 – 6900 Graybar Road
Richmond, BC
V6W 0A5 , Canada
Tel: +1 604 241 8152

Össur Nordic

P.O. Box 67
751 03 Uppsala, Sweden
Tel: +46 1818 2200
info@ossur.com

Össur Iberia S.L.U

Calle Caléndula, 93 -
Miniparc III
Edificio E, Despacho M18
28109 El Soto de la Moraleja,
Alcobendas
Madrid – España
Tel: 00 800 3539 3668
orders.spain@ossur.com
orders.portugal@ossur.com

Össur Europe BV – Italy

Via Baroaldi, 29
40054 Budrio, Italy
Tel: +39 05169 20852
orders.italy@ossur.com

Össur APAC

2F, W16 B
No. 1801 Hongmei Road
200233, Shanghai, China
Tel: +86 21 6127 1707
asia@ossur.com

Össur Australia

26 Ross Street,
North Parramatta
NSW 2151 Australia
Tel: +61 2 88382800
infosydney@ossur.com

Össur South Africa

Unit 4 & 5
3 on London
Brackengate Business Park
Brackenfell
7560 Cape Town
South Africa
Tel: +27 0860 888 123
infosasa@ossur.com



Instructions for Use




Össur hf.
Grjótháls 1-5
110 Reykjavík, Iceland
Tel: +354 515 1300
Fax: +354 515 1366



FEMALE PYRAMID INSERT FOR PRONG - LP

Product Number: A-145310

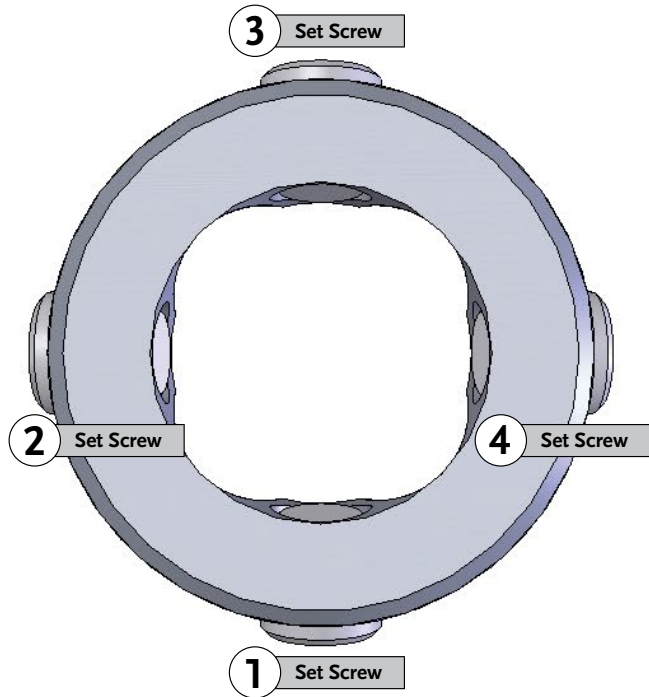

- Material: Titanium
- Weight limit: 166kg
- Weight: 33g

	Torque	Loctite
Set Screws	15 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.

Note: Angle adjustments are limited to max 2° in each direction. Ensure the set screws **DO NOT** interrupt the engagement of the threaded adapter (M36).

Note: When tightening the set screws use the following tightening sequence: 1 - 3 - 2 - 4. When final alignment has been decided tighten the screws first to 7 Nm (sequence 1 - 3 - 2 - 4) and then to 15 Nm (sequence 1 - 3 - 2 - 4).

ROTATION ADJUSTMENT:

Always engage threads as far as possible whilst maintaining correct rotation and torque pinch bolt.

- 2 year limited warranty

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.