

**Össur Americas**  
27051 Towne Centre Drive  
Foothill Ranch, CA 92610, USA  
Tel: +1 (949) 382 3883  
Tel: +1 800 233 6263  
ossurusa@ossur.com

**Össur UK Ltd**  
Unit No 1, S:Park  
Hamilton Road  
Stockport SK1 2AE, UK  
Tel: +44 (0) 8450 065 065  
ossuruk@ossur.com

**Össur Europe BV**  
De Schakel 70  
5651 GH Eindhoven  
The Netherlands  
Tel: +800 3539 3668  
Tel: +31 499 462840  
info-europe@ossur.com

**Össur Deutschland GmbH**  
Augustinusstrasse 11A  
50226 Frechen  
Deutschland  
Tel: +49 (0) 2234 6039 102  
info-deutschland@ossur.com

**Össur Canada**  
2150 – 6900 Graybar Road  
Richmond, BC  
V6W 0A5 , Canada  
Tel: +1 604 241 8152

**Össur Nordic**  
Box 770  
191 27 Sollentuna, Sweden  
Tel: +46 1818 2200  
info@ossur.com

**Össur Iberia S.L.U**  
Calle Caléndula, 93 -  
Miniparc III  
Edificio E, Despacho M18  
28109 El Soto de la Moraleja,  
Alcobendas  
Madrid – España  
Tel: 00 800 3539 3668  
orders.spain@ossur.com  
orders.portugal@ossur.com

**Össur Europe BV – Italy**  
Via Baroaldi, 29  
40054 Budrio, Italy  
Tel: +39 05169 20852  
orders.italy@ossur.com

**Össur APAC**  
2F, W16 B  
No. 1801 Hongmei Road  
200233, Shanghai, China  
Tel: +86 21 6127 1707  
asia@ossur.com

**Össur Australia**  
26 Ross Street,  
North Parramatta  
NSW 2151 Australia  
Tel: +61 2 88382800  
infosydney@ossur.com

**Össur South Africa**  
Unit 4 & 5  
3 on London  
Brackengate Business Park  
Brackenfell  
7560 Cape Town  
South Africa  
Tel: +27 0860 888 123  
infosa@ossur.com



Instructions for Use




**Össur hf.**  
Grjótháls 1-5  
110 Reykjavík  
Iceland



FEMALE PYRAMID INSERT  
FOR PRONG - LP

Product Number: A-145310

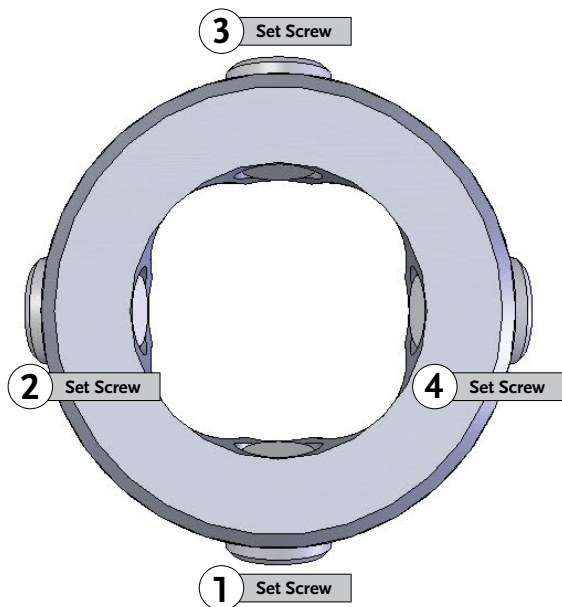

- Material: Titanium
- Weight limit: 166kg
- Weight: 33g

	Torque	Loctite
Set Screws	15 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.

**Note:** Angle adjustments are limited to max 2° in each direction. Ensure the set screws **DO NOT** interrupt the engagement of the threaded adapter (M36).

**Note:** When tightening the set screws use the following tightening sequence: 1 - 3 - 2 - 4. When final alignment has been decided tighten the screws first to 7 Nm (sequence 1 - 3 - 2 - 4) and then to 15 Nm (sequence 1 - 3 - 2 - 4).

#### ROTATION ADJUSTMENT:

Always engage threads as far as possible whilst maintaining correct rotation and torque pinch bolt.

- 2 year limited warranty

Rated to 166 kg/366 lbs, for normal use.

ISO 10328 - P7 - 166 kg \*)



\*) Body mass limit not to be exceeded!



For specific conditions and limitations of use see manufacturer's written instructions on intended use!

**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.