# Instructions for use

## Atlas® Collar

*Life Without Limitations®*

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## **COLLAR SIZING AND APPLICATION**

Proper application of the Collar is as important as proper sizing for immobilization and patient comfort. The two-piece design consists of a FRONT piece and BACK piece which are packaged as a set. Always maintain the patient's head in neutral alignment. These directions are guidelines only; follow a qualified healthcare provider's instructions.

1. After selecting the properly sized Collar, apply the FRONT piece of the Collar to the front of the patient's neck. Center the Collar. The front arrow should point upward.
2. Center the Collar BACK piece on to the patient's neck. With the FRONT piece properly secured, use the bilateral Velcro straps to fasten the FRONT and BACK pieces together.
3. Improvement of immobilization and comfort is achieved by minor adjustments of the straps and BACK piece supports.

## **REMOVAL**

Before removing the Collar, mark the position of the Velcro strap location to ensure re-application maintains the proper fit. With the Collar secured on the patient, unfasten the Velcro strap on the RIGHT side and then the LEFT side. Remove the Collar from the patient.

## **ALTERNATIVE SIZING METHOD**

Proper sizing of the Collar is important for patient immobilization and comfort. Always maintain the patient's head in neutral alignment. These directions are guidelines only; follow a qualified healthcare provider's instructions.

### **Height:**

1. Measure the vertical distance from the top of the patient's shoulder to the tip of the chin (Figure 1. "A").
2. The measurement corresponds to the HEIGHT of the Collar. If the patient's measurement falls between two consecutive sizes, apply the smaller size first.

### **Circumference:**

3. Measure around the patient's neck (Figure 2. "B").
4. The measurement corresponds to the size CIRCUMFERENCE of the Collar. If the patient's measurement falls between two consecutive sizes, apply the larger size first.

## **ALTERNATIVE SIZING METHOD**

Use your fingers to measure the vertical distance from the top of the patient's shoulder to the tip of the chin. Position your fingers against the FRONT side of the Collar from the plastic bottom edge to the top of the Velcro hook fastener. Select the size closest to the height of your fingers.

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### **Össur Iberia S.A.**