# Instructions for use

Rehab

*Life Without Limitations®*

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	6
ⓕⓗ Guide de fabrication.....	7
ⓔⓢ Instrucciones para el uso.....	8
ⓔⓣ Istruzioni per l'uso.....	9
Ⓝⓞ Bruksanvisning.....	10
ⓓⓐ Brugsanvisning.....	11
Ⓢⓕ Bruksanvisning.....	12
ⓕⓙ Käyttöohjeet.....	13
Ⓝⓕ Gebruiksaanwijzing.....	14
ⓔⓟ Instruções de Utilização.....	15

**EN - Caution:** This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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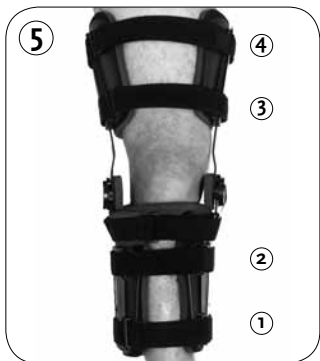
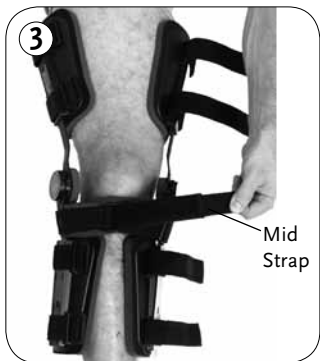
---

## CAUTION

This product is to be used only under the supervision of a licensed health care professional. Care must be taken to follow the advice of the health care professional when using this or any medical device.

## ADJUST THE BRACE LENGTH

- 1 Each pair of shells, (2 upper and 2 lower), must be set to the same position (**Figure 1**). Failure to do so could cause hinge malalignment and brace slippage.
- 2 Depress button on each shell and slide shell to required length (**Figure 2**).

## APPLY THE BRACE

- 3 Undo mid-strap and place brace under patient's leg (**Figure 3**). Attain correct hinge position by adjusting the mid-strap and pad.
- 4 Hinges are centered halfway between the front and back of the knee (**Figure 4**).
- 5 Hinges and shells are parallel (**Figure 5**).
- 6 Pass strap through the d-ring and fasten the hook tab end back onto the strap to secure (**Figure 3**).
- 7 The mid-strap should be tightened as firmly as possible without restricting circulation. Adjust strap length through hinge arm d-rings  
Correct Brace Position.

## ADJUST THE SHELL STRAP LENGTH

- 8 Align calf shells so hinges are parallel to tibia. Secure strap 1 (**Figure 5**). Pull strap tight, pass strap through d-ring and fasten the hook tab end back onto the strap.
- 9 Align thigh shells so hinges are parallel with the femur.
- 10 Secure straps 2,3, and 4 (**Figure 5**) using same procedure.
- 11 When desired strap length is achieved remove hook cover on shell and mid-strap (**Figure 6**). Press strap against hook to secure.
- 12 After straps are secured, check brace position again and adjust if necessary. Leave straps fastened at hook contact along shells. The brace is now properly sized for the patient. Future application and correct positioning can be done easily by fastening straps along the front of the brace position.

## RANGE OF MOTION

- 13 Push and set to desired extension position (**Figure 7**).
- 14 Push and set to desired flexion position (**Figure 7**).
- 15 After Range of Motion is set push Range Of Motion Spring Locks into place (**Figure 8**).
- 16 Make adjustments to both hinges.

**NOTE:** Range of Motion Lock is not to be used in the 105 degree

## CARE AND MAINTENANCE

Liners may be hand washed with cold water. Hang to dry.

## WARRANTY

Ossur post operative bracing product are disposable, limited use devices not intended for long term use. We warrant our products against defects in material and workmanship in the following manner; 30 days for skin contact parts (foam and straps) and 90 days for shells and hinges. Since we cannot control the manner in which such devices are fit, applied, adjusted or used, this warranty does not apply to products which have been neglected, abused, incorrectly applied or incorrectly fit. There are no warranties, either expressed or implied, other than those listed above.

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## Össur Iberia S.A.