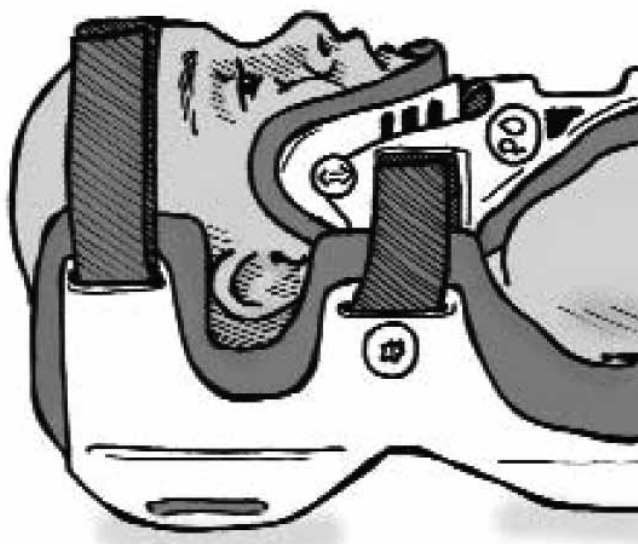
Instructions for use

Papoose®

*Life Without Limitations®*

.....	5
ⓔⓃ Instructions for use.....	8
ⓓⓔ Gebrauchsanweisung.....	11
ⓕⓗ Guide de fabrication.....	15
ⓔⓢ Instrucciones para el uso.....	19
ⓔⓣ Istruzioni per l'uso.....	23
Ⓝⓞ Bruksanvisning.....	27
ⓓⓐ Brugsanvisning.....	30
Ⓢⓕ Bruksanvisning.....	33
ⓕⓙ Käyttöohjeet.....	36
Ⓝⓕ Gebruiksaanwijzing.....	39

**EN - Caution:** This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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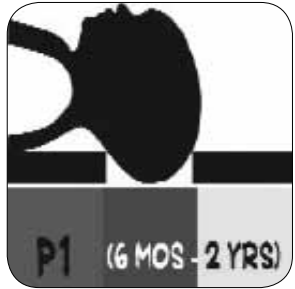
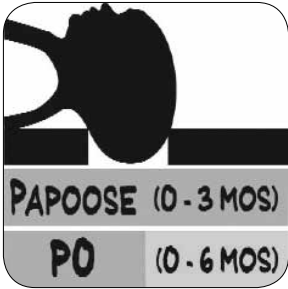
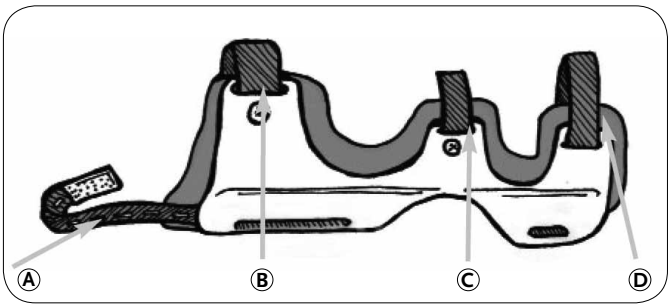
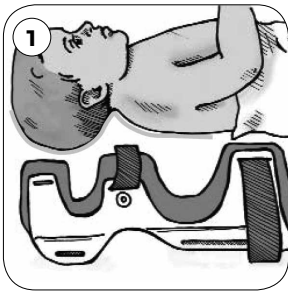
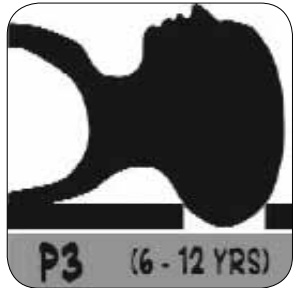
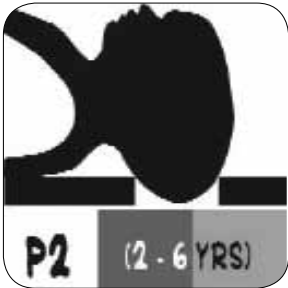
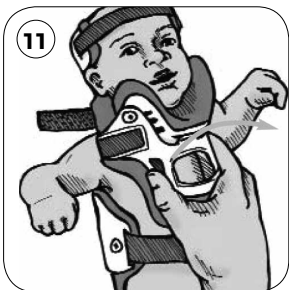
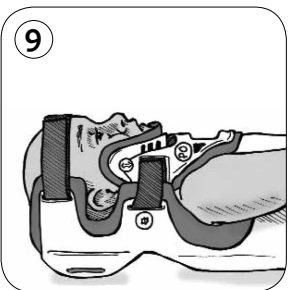
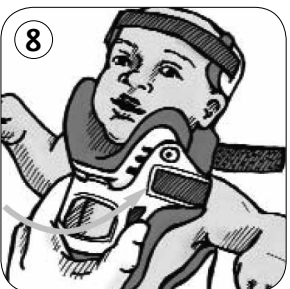
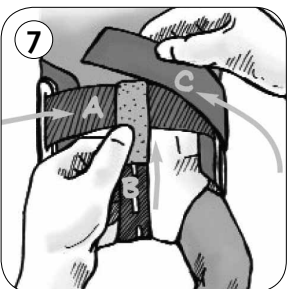
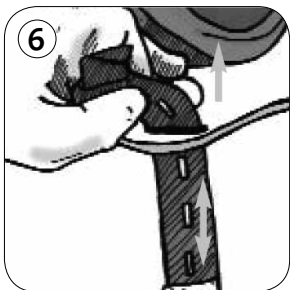
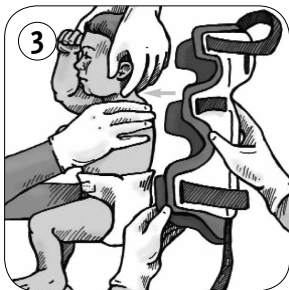
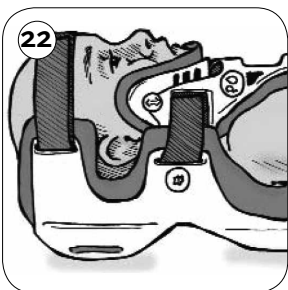
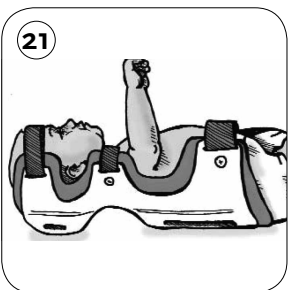
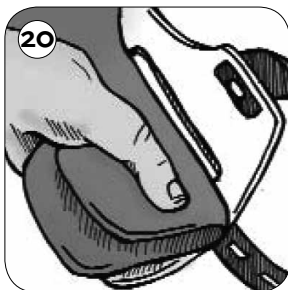
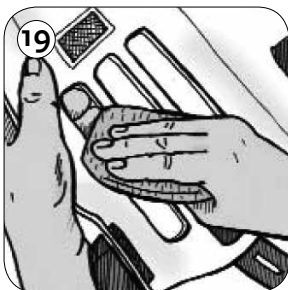
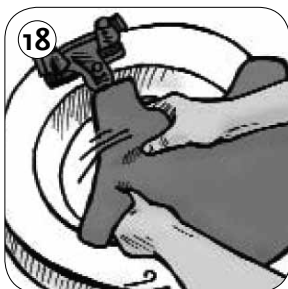
# ENGLISH

---

The Papoose Infant Spine Immobilizer is specifically designed for infants: our smallest, most important patients. This spinal stabilization device is engineered to meet the unique anatomical and physiological needs of Broselow Gray infants (birth to three months). For continued pediatric spinal care, Ossur offers the Miami Jr.® collars, Occian® AirWay PAD™ and Lil' Angel® Pediatric Halo. Please contact Ossur at 1-800-257-8440 for more information about these distinctive pediatric products.

**Caution:** Federal law restricts this device to sale by, or on the order of, a physician.

A Latex-Free Product

## INTRODUCTION (Figure 1)

Your child's doctor has determined that wearing a Papoose Infant Spinal Immobilizer will aid in rehabilitation by maintaining the spine and airway in the proper alignment while healing. The Papoose is designed to accommodate the relatively large size of an infant's head. True to Ossur's patented AirWay design, the rigid plastic brace also allows for proper airway alignment, even in the supine position. This pamphlet will explain the things you and your child's caregivers should do to properly care for your infant while in the Papoose, both at the hospital and at home. You can help your infant's healing process by following the nurse and doctor's instructions and the guidelines in this pamphlet.

## USEFUL TERMINOLOGY

- A. Leg (Perineal) Strap
- B. Waist Strap
- C. Collar Strap
- D. Head Strap

## APPLICATION

Two or three people are needed for initial Papoose placement: at least one to maintain the infant's head, neck and airway in proper alignment; and one to apply and fit the brace.

1. Using proper c-spine precautions, position infant with arms to the side, shoulders down and head aligned centrally (nose over belly button) (Figure 2).

The Papoose's shape accommodates the occipital offset and maintains correct airway and spinal alignment. The Papoose also protects against positional plagiocephaly (flattened head).

2. Carefully log roll the infant and center the Papoose behind the head and torso (Figure 3).
3. Roll the infant into the Papoose. Make sure that the blue Sorbatex™ padding extends beyond all plastic edges (Figure 4).
4. Apply the head strap. Feed the strap through the slot on the opposite side of the infant's forehead and then back upon itself to attach end to Velcro® fastener (Figure 5). The length of leg strap is adjustable to accommodate varying body lengths. To adjust, lift up blue pad at bottom of the brace and unbutton bottom strap. Refasten using the appropriate hole for your infant's body length (Figure 6).
5. Secure the leg and waist straps in three steps: (A) Velcro hook Waist Strap; (B) Leg Strap: attach Velcro loop to hook on Waist Strap; (C) Velcro loop Waist Strap. In this way the scratchy hook is always facing away from baby's tender skin (Figure 7).

## **COLLAR APPLICATION (OPTIONAL)**

For additional stability, the Papoose may be used in conjunction with the Miami Jr.® Po size Front.

Infants do not have the muscular strength required to lift their head. With physician's approval, **it may not be necessary to keep the front of the collar and/or the head strap in place while the infant is supine in bed.** In the supine position, the brace (with or without the head strap) will maintain proper spinal and airway alignment and limit head tilt and rotation. This will allow greater access to the child and optimize comfort. Replace the Front of the collar for transport, handling, or if the child is restless or agitated.

6. To apply the Miami Jr®, scoop the collar up under infant's chin. Sides of the collar Front should be oriented up, toward the ears (Figure 8).
7. While holding the front of the Miami Jr®. securely, curl the ends snugly against the infant's neck. Apply the collar straps. Tighten straps alternately, one at a time, to an equal length on both sides. Place the collar front inside the Papoose (Figure 9).
8. Velcro® straps must be aligned symmetrically and oriented "blue-on-blue" to the front adhesive sections. When the collar is properly fit, there should be equal amounts of excess Velcro® over-hanging the front blue adhesive sections. These may be trimmed if desired (Figure 10).

Collar application must be secure to ensure proper fit, maintain alignment and to prevent the chin from slipping inside. If the infant can slip his/her chin inside the collar, it is a clear indication that it is not snug enough.

## **ROUTINE CARE**

Keeping your infant's Papoose and the skin beneath it clean is an important part of treatment. Daily cleaning will help to prevent skin irritation. Cleaning the brace will require at least one set of replacement pads.

While at home, your infant should be carefully bathed with the Papoose on. After bathing, you will need to remove the brace (and collar) to wash his/her neck and back, and to change the pads.

### **Unless otherwise specified by your child's doctor:**

Do not remove the Papoose except to wash under it and to change the pads. Please note any special care instructions on the back of this pamphlet with important contact numbers.

## **BRACE REMOVAL & SKIN CARE**

Your infant must be lying down in proper anatomical (airway and c-spine) alignment to remove the brace. Two people will be required for this, one to maintain appropriate positioning, and one to clean the brace and child. Before removing the collar (if supplied), note or mark the ends of the Velcro straps with pen. When the collar is replaced, the straps should be returned to the same position. Undo the collar straps, and remove the collar (Figure 11).

1. With infant on his/her back, undo the waist and leg straps. This is also the appropriate position for diaper changes. With the bottom straps undone (top and neck straps remain fastened), the Papoose allows for easy access for necessary changing (Figure 12).
2. Undo the Head Strap. Using a washcloth, gently wash under the infant's chin, the chest and behind the ears with mild facial soap and water and completely dry the skin. Look for any redness or irritation under the collar. Clean the collar Front and exchange the

soiled pads with clean ones (Figure 13).

3. If available, hold the collar Front in place and gently roll the infant onto his/her side, and slide the Papoose out. Clean the Papoose and replace the soiled pads with clean ones as directed in "Papoose Cleaning" section (Figure 14).
4. While maintaining the head and body positioning, carefully wash the back of the infant's neck, and the back. Look for any redness, especially on the occiput (back of the head) (Figure 15).
5. Slide the Papoose carefully behind the infant's neck and back. Roll the infant back into the Papoose and securely refasten the collar and all straps (Figure 16).

## PAPOOSE CLEANING

These instructions are also appropriate for cleaning the Miami Jr®. Call Ossur Customer Service (1-800-257-8440) to purchase extra pads.

1. Remove the soiled blue pad. Look carefully at the shape as you remove it so that you can reposition the clean pads properly (Figure 17).
2. Wash pads with mild facial soap and water. **DO NOT use bleach or harsh detergents.** Thoroughly rinse pads with clean water. Wring out excess water and squeeze pads in a towel. Lay pads out flat to air dry. It should take less than 60 minutes for them to dry (Figure 18).
3. Wipe plastic shell clean with mild soap and water (Figure 19).
4. Attach new pads. Fold pads in half with dull side out next to the Velcro (shiny side goes against skin); then center pad in the white shell. Adjust pads as needed to make sure be resting on infant's clavicles nor digging into shoulders (Figure 20).

## FINAL CHECKLIST (Figure 21)

A properly applied Papoose will look like:

- No plastic touching skin. Blue Sorbatex pads extend beyond all plastic edges.
- No slack or gaps in any of the straps.

(Optional) (Figure 22)

- Sides of the Papoose should overlap the sides of the front.
- Collar extends from jaw to just below sternal notch.
- Chin is centered comfortably in the chin piece. Chin should not extend over edge of Sorbatex pad, nor fall inside collar.
- Velcro straps oriented blue-on-blue, both the same length.
- Front of collar angled up toward ears. Lower plastic edge should not be resting on infant's clavicles nor digging into shoulders.

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Papoose® is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.
### **Össur Iberia S.A.**