Instructions for use

Female Pylon Kit

Life Without Limitations®

ⓔⓃ	Instructions for use	3
ⓓⓔ	Gebrauchsanweisung	4
ⓕⓗ	Notice d'utilisation	5
ⓔⓈ	Instrucciones para el uso	6
ⓓⓐ	Brugsanvisning	7
ⓈⓋ	Bruksanvisning	8
ⓃⓁ	Gebruiksaanwijzing	9
ⓔⓓ	Instruções de Utilização	10

ENGLISH

A-762121 & A-762111 (Aluminium), A-766122, A-766112, A-766121, A-766111 (Carbon / Titanium)

Part number: A-762121 , A-762111

Contents

Female Pylon
Female Pyramid Tube Clamp

- Material: Aluminium
- Weight limit: 100 kg
- Length: 400mm / 250mm

Part number: A-766122, A-766112, A-766121, A-766111

Contents

Female Pylon
Female Pyramid Tube Clamp
Carbon Tube Insert
Male Pyramid Socket Adapter (A-766122, A-766112)

- Material: Carbon / Titanium
- Weight limit: 166 kg
- Length: 400mm / 250mm

	Torque	Locite
Pinch Bolt	10 Nm x 2	242
Set Screws	15 Nm	242

NOTE: Pinch bolt must be positioned anteriorly on the prosthesis. Ensure tube is fully inserted into tube clamp adapter before securing.

Procedure for cutting and assembling pylon:

The pylon should be shortened based on the Individuals measurements. This must be done according to the instructions below, avoid damaging the pylon as this may prevent easy insertion into the tube clamp adapter.

- I. Do not clamp tube directly in vice, use either 30mm-tube clamp or tube cutter.
- II. Ensure tube is cut straight with a hacksaw, not at an angle.
- III. Remove all burs with a file and add Carbon Tube Insert before inserting into tube clamp adapter.
- IV. Loosen pinch bolt on tube clamp adapter before inserting pylon.
- V. Ensure tube is fully inserted and in firm contact with base of tube clamp adapter.

- 2 year limited warranty.

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

Aluminium tested to ISO10328, load level A100, rated to 220lbs / 100kg for normal use.

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EN – Caution: Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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Össur Iberia S.A.