## ENGLISH

- Material: Aluminum
- Weight limit: 100kg
- Weight: 52g

	Torque	Loctite
Mounting Screws	10 Nm	243
Pinch Bolt	10 Nm	243

Loctite 243 is used on the mounting screws.

For secure fastening use the loctite recommended and apply rated torque twice.

Loctite 243 is used on the pinch bolt.

Pinch Bolt must be positioned anteriorly on the prosthesis. Ensure tube is fully inserted into Tube Clamp Adapter before torquing.

Loctite 243 is used on the pinch bolt.

- 2 year limited warranty**

Loctite 243 is used on the mounting screws.

Tested to ISO10328, load level A100 + 20%, rated to 100kg / 220lbs, is not recommended for active use.

Loctite 243 is used on the mounting screws.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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**EN – Cautión:** Össur productos e componentes de acuerdo a tested and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2017 IFU 0051 1152-001 Rev. 6