ENGLISH

- Material: Titanium
- Weight limit: 166kg
- Weight: 78g

	Torque	Loctite
Pinch bolt	10 Nm	243
Set Screws	15 Nm	243

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

For secure fastening use the loctite recommended and apply rated torque twice.

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

Pinch Bolt must be positioned anteriorly on the prosthesis. Ensure tube is fully inserted into Tube Clamp Adapter before torquing.

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

- 2 year limited warranty**

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Össur products or components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2017 IFU 0080 1161_001 Rev. 6