## ENGLISH

**AJ-242100**

- Material: Aluminum
- Weight Limit: 45kg
- Weight: 34g

	Torque	Loctite
Mounting Screws	6 Nm	243
Set Screws	4 Nm	243

Loctite 242100, 243 and 243 screws

For secure fastening use the loctite recommended and apply rated torque twice.

Loctite 242100, 243 and 243 screws

- 2 year limited warranty**

Loctite 242100, 243 and 243 screws

Aluminum tested to ISO 10328, load level A60, rated to 100lbs/45kg for normal use.

Loctite 242100, 243 and 243 screws

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with this standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2017 IFU 0190 1178\_001 Rev. 4