<ul style="list-style-type: none">Material: Aluminium/Stainless Weight limit: 45kg Weight: 105g		
	Torque	Loctite
Foot Bolt	20Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.

Note: A bonding agent may be required to secure the adapter to the foot, depending on the SACH foot selected. If in doubt contact the SACH foot manufacturer.

It is essential that only the supplied Foot Bolt is used to secure the SACH Foot Adapter in position, providing a secure thread engagement. If necessary the Foot Bolt should be cut to length.

- 2 year limited warranty

Aluminum tested to ISO10328, load level A60, rated to 100lbs/45kg for normal use.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Os produtos e componentes are tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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JUNIOR S.A.C.H. FOOT ADAPTER