



Instructions for Use

QUICK-ALIGN™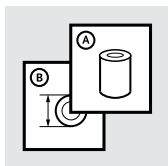

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	6
FR	Notice d'utilisation	8
ES	Instrucciones para el uso	10
IT	Istruzioni per l'uso	12
NO	Bruksanvisning	14
DA	Brugsanvisning	16
SV	Bruksanvisning	18
EL	Οδηγίες Χρήσης	20
FI	Käyttöohjeet	22
NL	Gebruiksaanwijzing	24
PT	Instruções de Utilização	26
JA	取扱説明書	28
ZH	中文说明书	30
KO	사용 설명서	32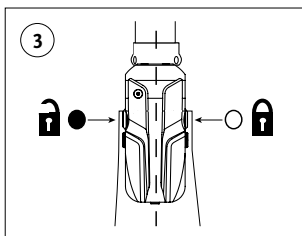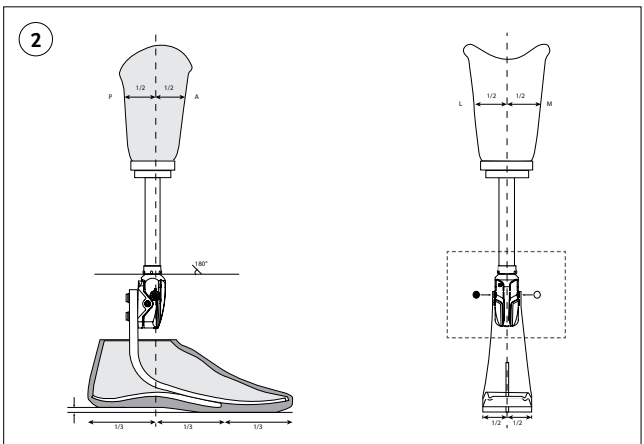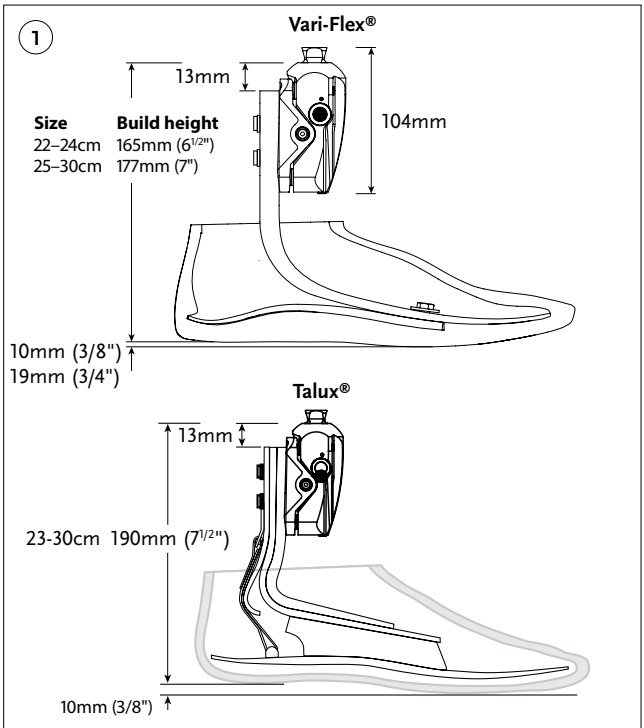

ENGLISH

- Adjustable heel height adapter for use with Talux and Variflex.
- For low to moderate impact levels.
- Weight limit: 125 kg (275 lbs).
- 8° of heel height adjustment (30 mm or 1¼") with size 25 foot.
- Aluminum body.
- Male pyramid attachment.
- This product is to be used exclusively for lower extremity.

DIMENSIONS

Figure 1

ATTACHING TO FOOT MODULE

To attach adapter, use correct sized bolts for foot category.

- Place washers on foot module, rounded side facing foot.
- Insert fastening bolts through washers into foot module.
- Apply Loctite 410 to entire length of bolt threads and torque as follows:

Foot	Cat.	Bolt size	Length	Torque
Vari-Flex	1–4	M8	20 mm	22 Nm (16 ft-lbs)
Vari-Flex	5–6	M8	25 mm	22 Nm (16 ft-lbs)
Vari-Flex	7–9	M10	25 mm	34 Nm (25 ft-lbs)
Talux	1–4	M8	25 mm	22 Nm (16 ft-lbs)
Talux	5–6	M8	30 mm	22 Nm (16 ft-lbs)
Talux	7–8	M10	30 mm	34 Nm (25 ft-lbs)

BENCH ALIGNMENT

Using a 10 mm heel height, adjust pyramid so that its top surface is horizontally level (**Figure 2**).

- See alignment instructions for foot module for information regarding overall alignment.

PUSH BUTTON MECHANISM (**Figure 3**):

- Press push button to unlock adapter (black button).*
- Adjust adapter for heel height using body weight or by hand.
- Once adapter is correctly adjusted, return push button to locked position (white button).

* Do not use the adapter for walking while it is unlocked.

Note: The push button may become stiff when the adapter is not used for a period of more than a day or two. Take care unlocking the adapter when the button is stiff.

Note: Fixing an extension to the push button may result in accidental release.

CARE AND CLEANING

- Avoid exposing adapter to extreme heat, sand, water or solvents.
- Clean adapter with a soft, damp cloth.

Do not use solvents or compressed air to clean adapter.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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