






Instructions for Use

MIAMI J[®] ADVANCED

		3
EN	Instructions for Use	4
DE	GEBRAUCHSANWEISUNG	6
FR	NOTICE D'UTILISATION	8
ES	INSTRUCCIONES PARA EL USO	10
IT	ISTRUZIONI PER L'USO	12
DA	BRUGSANVISNING	14
SV	BRUKSANVISNING	16
EL	Οδηγίες χρήσης	18
FI	KÄYTTÖOHJEET	20
NL	GEBRUIKSAANWIJZING	22
PT	INSTRUÇÕES DE UTILIZAÇÃO	24
PL	Instrukcja użytkowania	26
JA	取扱説明書	28
ZH	中文说明书	30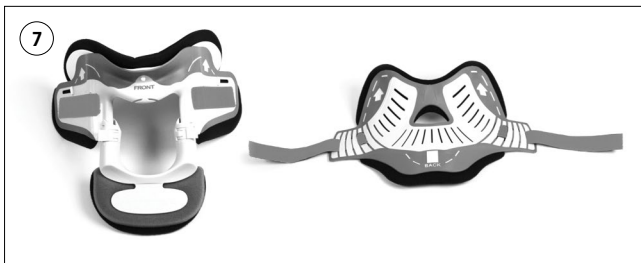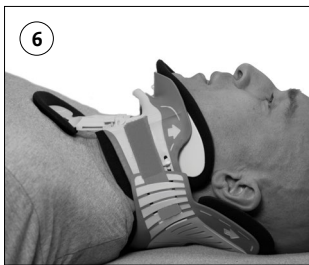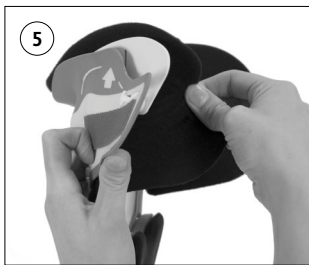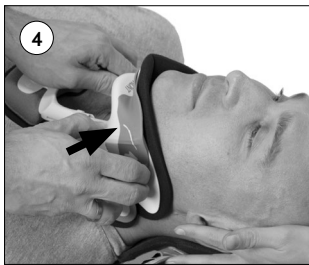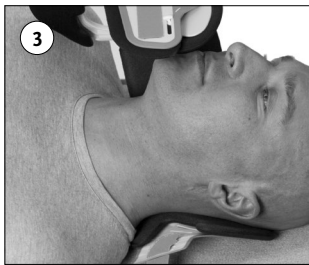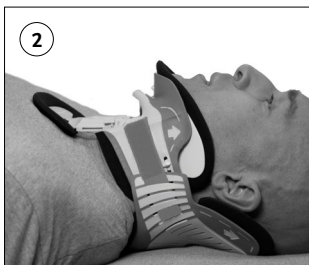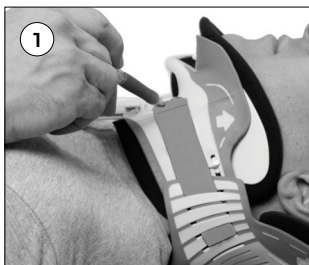

ENGLISH

INTRODUCTION

These instructions do not supercede the specific orders of your physician. Your doctor has determined that wearing a Miami J® Advanced collar will aid in your rehabilitation. The collar will maintain your neck in the proper position while it is healing. Not being able to turn or shake your head “yes” or “no” may seem unusual, but limiting the motion of your neck is an important part of your treatment. This pamphlet will explain the things you should do to properly use and care for your Miami J® Advanced collar. You can help the healing process by following your doctor’s instructions and the guidelines in this pamphlet.

PRECAUTIONS

- You must be lying flat to remove the collar unless your doctor gives you permission to do this in a sitting position.
- You will need a second person to help you when cleaning the collar.
- Unless otherwise specified by your doctor: **Do not change the height adjustment.** Do not remove the collar except to wash under it and change the pads.
- **Collar removal should be done with physician permission only.**

Not made with natural rubber latex

SKIN CARE

Keeping your collar and the skin beneath it clean is an important part of your treatment. Daily cleaning will help to prevent skin irritation. A second person must assist during this procedure.

1. Before removing the collar, mark both sides where the straps overlap the front of the collar (**Figure 1**).
2. Lie flat on your back on a bed without a pillow, arms to the side, shoulders down and head aligned centrally (**Figure 2**).
3. Keep your head and neck in a neutral position at all times. Do not bend your neck forward, backward, or from side to side.
4. Detach the strap on one side of the collar and carefully slide the collar out from under neck (**Figure 3**).
5. Wash the front and back of neck with mild facial soap and water and completely dry the skin. Observe for any redness or irritation under the collar, especially over bony areas like the chin, clavicles, or back of the head. Contact your physician if irritation is present.
6. Clean the collar as described in the **Collar Cleaning** instructions below.
7. Replace the collar by sliding the back carefully behind the neck making sure it is centered behind the head. Long hair should be placed outside the collar.
8. Slide the front up under the chin, aligning the front edge of the collar’s chin support with the front of the chin (**Figure 4**).
9. While holding the front securely, curl the loose side snugly against the neck and securely apply the strap to the front. The end of the strap should be placed according to the mark on the collar made prior to removal (**Figure 1**).

CHECKLIST

- Chin is centered comfortably in the chin piece.
- The front and back openings in the collar should be centered on the neck.
- Make sure the collar is nicely snug on the chin. You should not be able to slip your chin inside the collar.
- The plastic edges of the collar should not be touching the skin. Make sure the blue pads extend beyond all plastic edges of the collar (**Figure 6**).

COLLAR CLEANING

- Remove the pads from the plastic parts (**Figure 5**).
- Wash the pads with mild soap and water. **Do not use bleach or harsh detergents. Do not machine wash.**
- Thoroughly rinse the pads with clean water.
- Wring out the excess water and squeeze in towel.
- Lay flat to dry, which should take less than 60 minutes.
- Wipe the plastic collar shell clean with mild soap and water.
- Replace pads by attaching the dull side to the Velcro on the inside of the collar (**Figure 7**).

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