


# CASTING AND MODIFICATION FORM

## Iceross® Seal-In V

Patient Name: \_\_\_\_\_ CP/CPO: \_\_\_\_\_ Date: \_\_\_\_\_

Limb: R L Liner Size: \_\_\_\_\_ Össur Foot: \_\_\_\_\_ Foot Size: \_\_\_\_\_ Weight: \_\_\_\_\_ Impact: \_\_\_\_\_

Notes: \_\_\_\_\_

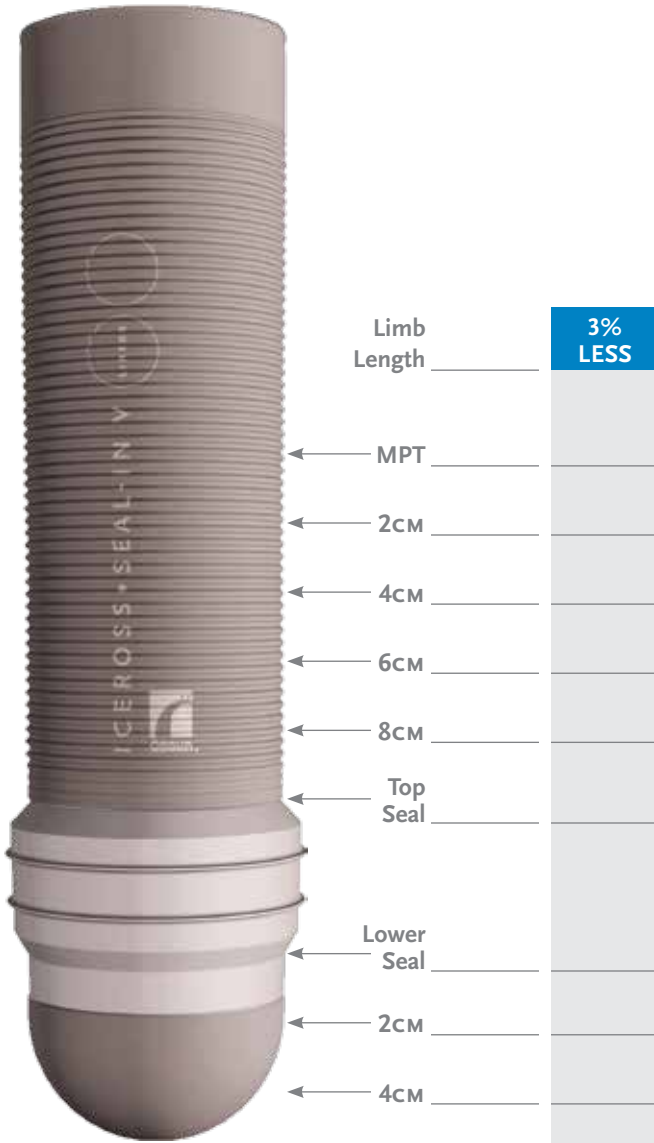

### Össur Casting Recommendations

- Neutral hand cast with no initial shaping or pressure during casting.
- Cast with elastic plaster bandage then rigid plaster bandage.
- Wrap proximal to distal with a 3-5° knee flexion.

— OR —

- Use CAD digital system with a 3-5° knee flexion.

### Össur Modification Recommendations

- Shave off the seal to be flush with the positive mold.
- Reduce 3% globally.

### REDUCTION CHART

Meas. (cm)	3% Reduction	Meas. (cm)	3% Reduction
16	15.5	28.5	27.7
16.5	16	29	28.1
17	16.5	29.5	28.6
17.5	17	30	29.1
18	17.5	30.5	29.6
18.5	18	31	30.1
19	18.4	31.5	30.6
19.5	18.9	32	31
20	19.4	32.5	31.5
20.5	19.9	33	32
21	20.4	33.5	32.5
21.5	20.9	34	33
22	21.3	34.5	33.5
22.5	21.8	35	34
23	22.3	35.5	34.4
23.5	22.8	36	34.9
24	23.3	36.5	35.4
24.5	23.8	37	35.9
25	24.3	37.5	36.4
25.5	24.7	38	36.9
26	25.2	38.5	37.4
26.5	25.7	39	37.8
27	26.2	39.5	38.3
27.5	26.7	40	38.8
28	27.2		

### ORDERING INFORMATION

#### SIZES

- 11 sizes (18 - 36cm)


#### OPTIONS:

- I-4713XX Seal-In V

#### PROFILE

- Wave 3mm

XX: 18, 20, 22, 23.5, 25, 26.5, 28, 30, 32, 34, 36

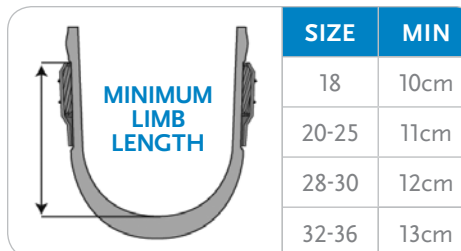


# CASTING AND MODIFICATION FORM

## Iceross® Seal-In V HP

Patient Name: \_\_\_\_\_ CP/CPO: \_\_\_\_\_ Date: \_\_\_\_\_

Limb: R L Liner Size: \_\_\_\_\_ Össur Foot: \_\_\_\_\_ Foot Size: \_\_\_\_\_ Weight: \_\_\_\_\_ Impact: \_\_\_\_\_

Notes: \_\_\_\_\_

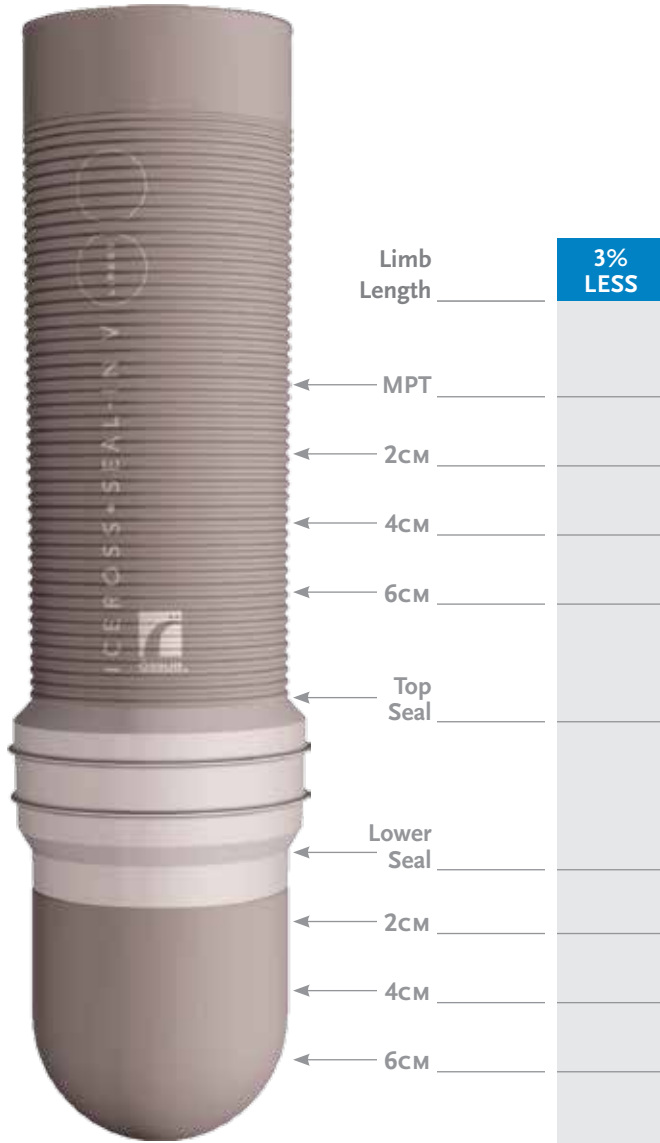

### Össur Casting Recommendations

- Neutral hand cast with no initial shaping or pressure during casting.
- Cast with elastic plaster bandage then rigid plaster bandage.
- Wrap proximal to distal with a 3-5° knee flexion.

— OR —

- Use CAD digital system with a 3-5° knee flexion.

### Össur Modification Recommendations

- Shave off the seal to be flush with the positive mold.
- Reduce 3% globally.

### REDUCTION CHART

Meas. (cm)	3% Reduction	Meas. (cm)	3% Reduction
16	15.5	28.5	27.7
16.5	16	29	28.1
17	16.5	29.5	28.6
17.5	17	30	29.1
18	17.5	30.5	29.6
18.5	18	31	30.1
19	18.4	31.5	30.6
19.5	18.9	32	31
20	19.4	32.5	31.5
20.5	19.9	33	32
21	20.4	33.5	32.5
21.5	20.9	34	33
22	21.3	34.5	33.5
22.5	21.8	35	34
23	22.3	35.5	34.4
23.5	22.8	36	34.9
24	23.3	36.5	35.4
24.5	23.8	37	35.9
25	24.3	37.5	36.4
25.5	24.7	38	36.9
26	25.2	38.5	37.4
26.5	25.7	39	37.8
27	26.2	39.5	38.3
27.5	26.7	40	38.8
28	27.2		

### ORDERING INFORMATION

#### SIZES

- 11 sizes (18 - 36cm)

#### OPTIONS:

- I-4723XX Seal-In V

#### PROFILE

- Wave 3mm

XX: 18, 20, 22, 23.5, 25, 26.5, 28, 30, 32, 34, 36

SIZE	MIN
18	13cm
20-25	14cm
28-30	15cm
32-36	16cm

