PORTUGUÊS

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EN - Caution: Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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For secure fastening use the loctite recommended and apply rated torque twice.

Cutting and assembly of Carbon Tube:

The tube should be shortened according to measurements. Avoid damage to tube as it may prevent easy insertion into Tube Clamp Adapter.

1. Remove Tube Insert.
2. Clamp tube with a 30mm cutting jig.
3. Ensure tube is cut straight with a fine tooth saw.
4. Remove all burrs with a file.
5. Replace Tube Insert before inserting into Tube Clamp Adapter.
6. Ensure tube is fully inserted and in firm contact with base of Tube Clamp Adapter.

2 year limited warranty.

Titanium tested to ISO 10328, load level A100+66%, rated to 366lbs/166kg for normal use.

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