

POWER KNEE™



ADDITIONAL FITTING INSTRUCTIONS FOR THE POWER KNEE.

Please observe the following guidelines for torquing the proximal pyramid set screws of the POWER KNEE:

- During POWER KNEE fitting, proximal pyramid set-screws should be torqued to 15Nm.
- Proximal pyramid torquing procedure should be repeated at the end of first day of training.
- Proximal pyramid torquing procedure should be repeated at the end of first week of knee operation.

Össur Head Office

Grjótháls 5
110 Reykjavík, Iceland
Tel: +354 515 1300
Fax: +354 515 1366
mail@ossur.com

Össur Americas

Tel: +1 (949) 382 3883
Tel: +1 800 233 6263
Fax: +1 800 831 3160
ossurusa@ossur.com

Össur Canada

Tel: +1 604 241 8152
Fax: +1 604 241 8153

Össur Europe

Tel: +800 3539 3668
Tel: +31 499 462840
Fax: +31 499 462841
info-europe@ossur.com

Össur Nordic

Tel: +46 1818 2200
Fax: +46 1818 2218
info@ossur.com

Össur Iberia S.A.

Tel: +34 91 636 06 93
Fax: +34 91 637 21 43
ortesica@ossur.com

Össur UK

Tel: +44 161 490 8500
Fax: +44 161 490 8501
ossuruk@ossur.com

Össur Asia

电话: +86 21 6127 1700
传真: +89 21 6127 1799
asia@ossur.com

Össur Asia-Pacific

Tel: +61 2 9630 9206
Fax: +61 2 9630 9268
info-asiapacific@ossur.com



Life Without Limitations