





Basic Operation

Function	Hold	Press	BEEP
POWER ON/OFF	 1x  1x		2 BEEPS=ON 3 BEEPS=OFF
Keypad unlock	 1x  1x		2 BEEPS= UNLOCKED
Set heel height	 1x	 2x	1x START 1x MOVE 1x COMPLETE

NOTE: Keypad will be disabled after 60 seconds of inactivity.



Basic Operation

Warning	Indicates
2× BEEP	Approx 120 min left
3× BEEP	Approx 60 min left
4× BEEP	Approx 30 min left
5× BEEP	Foot will return to neutral

PROPRIO FOOT®

Do not fit boots or shoes that cover the ankle area!

The PROPRIO FOOT® will not function properly if ankle motion is restricted in any way.

An alarm and vibratory warning will sound if the ankle gets stuck. Tapping lightly with heel and toe will free it up again.