



Instructions for Use

---

PHILADELPHIA® LINER

## DESCRIPTION

- Specifically designed for the Philadelphia® Cervical Collar to enhance the patient's level of comfort during periods of extended wear.
- Made of 83% polyester and 17% spandex for optimal comfort and absorption.
- Hygienic and Soft.
- Naturally wicking polyester is further treated with 3M Scotchgard Moisture Management finish
- Treated with Rudolf biocide Ruco-Bac Med.
- Form fitting sizes to ensure easy application.
- Washable and Reusable.
- Sizes available to form fit all Philadelphia® Cervical Collars.

## APPLICATION

Each Philadelphia® Liner is individually designed to fit securely on the Philadelphia® Cervical Collar. After matching the liner size to the corresponding collar, insert the collar into the liner as follows:

- Align the top end of the collar with the upper edge of the liner seam. Insert the collar into the liner (**Figure 1**).
- Simply follow around the edge of collar with the liner seam. The seam should finish at the opposite end of the collar (**Figure 2**).
- With both hands, smooth out any wrinkles and adjust position of Front Hole relative to tracheotomy opening. Complete by tucking the label under the liner (**Figure 3**).



- For proper fit, the liner label and central seam must be parallel to the plastic reinforcement. This application is identical for both the front and back portion of the Philadelphia® Cervical Collar.

## **SIZING**

Philadelphia® Liner sizes are identical to Philadelphia® Cervical Collar sizes. The liners fit both our Original and Tracheotomy designs.

### **Adult Sizes**

	2 ¼	3 ¼	4 ¼	5 ¼
X-Small/Small	PHP-LCS			
Medium	PHP-LCM			
Large	PHP-LCL			
Extra Large	PHP-LCXL			

*Pediatric and Infant sizes are no longer available.*

## **CARE INSTRUCTIONS**

- Philadelphia® Liner is washable and reusable.
- It is important to clean the liner of bodily fluids and contaminants after each use. Hand wash in warm water with mild soap. Line dry, in shade.
- Do not iron.
- Do not dry clean.

**Caution:** This product has been designed and tested as a single use product, and is not intended for multiple use. In addition to increasing the risk of infection, used product components may contain small unnoticeable defects that may lead to non-optimal performance, and even mechanical failure. If any problems occur with the use of this product, immediately contact your medical professional.

---

**Össur Head Office**



Grjótháls 5

110 Reykjavík, Iceland

Tel: +354 515 1300

Fax: +354 515 1366

mail@ossur.com

