1

2

3

4

## ENGLISH

- Open the package and remove the heel wedges and arch support (**Figure 1**).
- Remove the dorsal cover and liner from the inside of the Rebound Air Walker.
- To place the heel wedges, remove the adhesive backing from the adhesive on the bottom wedge and place the heel wedges into the heel of the Rebound Air Walker (**Figure 2**).  
**Note:** *If all wedges are not needed, individual wedges can be removed prior to initial placement in the Rebound Air Walker.*
- As needed, the arch support can be placed into the walker using the provided two-sided tape by attaching it first to the arch support and then to the appropriate position on the wedges (**Figure 3**). The arch support can be used as either a left or a right, simply by turning it over.  
**Note:**
  - It is recommended to determine the appropriate placement of the arch support prior to affixing it into position with the two-sided tape.*
  - It is also recommended to affix the arch support to the top wedge, so it remains in place as the lower wedges are removed.*
- During the course of therapy, individual wedges can be removed by removing the entire heel wedge assembly from the Rebound Air Walker and then separating unnecessary wedges from the assembly (**Figure 4**). Wedges are designed to be removed from the bottom wedge to the top wedge, in order to maintain the correct shape.  
**Note:** *Individual wedges are 10 mm in thickness.*

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