ENGLISH

INDICATION FOR USE

Mild to moderate unicompartmental osteoarthritis.

CAUTION

For patients with peripheral vascular disease, neuropathy and sensitive skin, physician supervision is recommended.

If redness appears, decrease usage of the brace and increase slowly as skin adapts to it.

PRODUCT OVERVIEW (Figure 1)

- a. Dynamic Force Strap (DFS)
- b. D-Ring
- c. Hinge
- d. Knee Cap Opening
- e-h. Tabs

POSITIONING (Figure 2)

- Place hinge medially for medial compartment osteoarthritis and laterally for lateral compartment osteoarthritis.
- Position brace with one and three dots above knee and two and four dots below knee.
- Hinge center should be slightly above mid-point of the patella.

DONNING (Figure 3-7)

- Sit on the edge of a chair. Undo contact closures and bend knee to 30° with foot flat on floor.
- Keep brace in place with one hand and with the other hand, fasten one dot tab firmly with contact closure. Do the same for two dot tab.
- Proceed with fastening three dot tab and then four dot tab.
- Check that all tabs are firmly but comfortably fastened and adjust if necessary.

DYNAMIC FORCE STRAP ADJUSTMENT (Figure 8 - 9)

- Thread the Dynamic Force Strap through the D-Ring and bend knee to an 80° angle with foot flat on floor
- Hold brace with one hand to prevent movement and tighten the Dynamic Force Strap with the other.
- Strap should be tightened until giving maximum pain relief.

POSITIONING (Figure 10)

Check that brace is correctly positioned and that it sits firmly on knee, providing desired pain relief.

MAINTENANCE

- Clean dirt from hinge using tap water.
- Avoid prolonged contact with salt and chlorinated water, rinse well in case it occurs.
- Hand wash using mild soap and rinse thoroughly. For machine wash, a wash bag is recommended and a delicate wash program.
- Air dry (no tumble dry).

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©Copyright Össur 2018