## ENGLISH

### INTENDED USE

The Nike Sole Junior version is intended to be used with Cheetah® Junior and Flex-Run™ Junior as a part of a exoskeletal prosthesis for lower limb amputations.

- Activity level: moderate – extreme
- Impact levels: Low - extreme
- Weight limit: See Cheetah Junior and Flex-Run Junior selection chart for specific foot weight limit

### SOLE ATTACHMENT

- For tighter fit of the sole or to reduce noise bond insert on lower surface of foot module using Loctite 495, clean bonding surfaces before applying adhesive, keep under pressure for 10 minutes cure (**Figure 1**).
- Slide the Nike sole on to the foot and make sure all the clips go over the foot (**Figure 2**).
- Pull the rubber strap of the sole over the medallion and slide it under. When detaching the sole, start by pulling the strap away from the medallion, then slide the sole off the foot (**Figure 3**).

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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© Copyright Össur 2015