## ENGLISH

**Note:** Refer to the SmartCast Tool instructions when making the plaster cast. If using an old cast without SmartCast Tool, use the Guidance Fixture (not shown) instead to enable accurate positioning of the Puck. Nail it to the distal end of the cast.

1. Trim the Distal Spacer Pad (1) to the size of the distal end of the cast, put it in place and pull the inner PVA bag (2) over.

**Note:** If using Pressure Pads, adhere them onto the PVA bag over the bony areas. Make sure that the PVA bag is completely dry.

2. Fill the heads of the four screws in the Puck (3) with clay.
3. Position the Puck on top of the cast pulling the PVA bag through the central hole.
4. Place the washer (4) from the SmartCast Tool (5) inside the PVA bag. Secure the Puck to the SmartCast Tool with the bolt (6). Make sure that the Puck is orientated with the screws in AP/ML position.
5. Tie off (7) the PVA bag and evacuate the air.
6. Fill the Puck with clay and seal it with sealing resin.
7. Tie on the lay-up material. Tie the first layer in the most proximal groove (8) and the subsequent layers gradually more distally.
8. Apply the outer PVA bag and place the Aluminum Ring inside it. Keep the ring 1-2cm (1/2"-1") above the Puck.
9. Pour resin, allowing it to impregnate around the Puck.
10. Push the Aluminum Ring down over the Puck.
11. Finish lamination.
12. Remove the bolt and the socket from the cast. Trim and smooth edges.
13. Mount the locking mechanism (ICELOCK Ratchet SP 125) according to the instructions enclosed.

- 1 year limited warranty

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©Copyright Össur 2017 IFU 0011 1220\_001 Rev. 5