ENGLISH

CONTENT

L-692022 Fabrication Ring w/o-ring **(fig. 6)**.
L-692110 Funnel **(fig. 2)**.
L-692020 Ratchet Lock w/o adapter**(fig. 3)**.
L-692027 Release Unit, long **(fig. 4)**.
L-192000 Attachment Pin ratchet, standard **(fig. 1)**.
L-693112 Silicone Seal **(fig. 7)**.
Items 6 and 7 are used when the socket is fabricated.

Use with L-680000 Fabrication Tooling (reusable).

Tools required

- 8mm (5/16) socket wrench.

INSTALLATION

- Place Ratchet Lock (3) into bottom of socket with release unit hole in correct position.
- Shorten Release Unit (4) to required length and screw it firmly and completely into hole on socket and Ratchet Lock (3) **OPTIONAL:** L-692016 Release Key (5) makes installation and removal of Release Unit (4) easier and is ideal when using prosthesis with exoskeletal design.
- Screw Funnel (2) into threads above Ratchet Lock (3) and torque to 8Nm (6ft.lb) using Cross Driver (9) (comes with Fabrication Tooling) and 8mm torque wrench.

ATTACHMENT PIN

Apply Loctite 410 or 411 onto threads on Attachment Pin (1) and screw it into Locking Liner. Torque Attachment Pin (1) to 4Nm (3ft.lb) with 7mm wrench.

If needed, L-192003 Attachment Pin ratchet short and L-192005 Attachment Pin ratchet long are available.

Note: Use Attachment Pins for ICELOCK 600/100 series only.

Important: For best results, ensure that the pin is fully engaged. If temporary swelling prevents full engagement, a minimum of four teeth must enter the lock for safety reasons.

Limited one-year warranty.
Rated up to high activity level (K3).
Distal/weight bearing adapter determines weight limit.

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

Other locking components available for 600 series:
L-631000 ICELOCK lanyard 631.
L-641000 ICELOCK valve 641.
L-651000 ICELOCK smooth lock 651.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2017 IFU 0148 IFU 1223_001 Rev. 6
Össur Deutschland GmbH