ENGLISH

CONTENT

L-692030 Stainless Pyramid 673 (rated up to 100kg / 220lbs) **(fig. 3a)**

OR

L-692135 Titanium Pyramid 672 (rated up to 166kg / 365lbs) **(fig. 3b)**

AND

L-692142 Pyramid Cap **(fig. 2)**

L-692032 Pyramid Bolt w/ o-ring **(fig. 1)**

L-693115 Silicone Ring **(fig. 11)**

L-693112 Silicone Seal **(fig. 6)**

Use with L-680000 Fabrication Tooling (reusable)

When fabrication is complete, insert one of the following locking components (include Fabrication Ring (4) needed in the fabrication process)

L-621000 ICELOCK ratchet 621

L-631000 ICELOCK lanyard 631

L-641000 ICELOCK valve 641

L-651000 ICELOCK smooth 651

Tools required

- 4mm and 5mm hexagonal keys
- 8mm socket wrench

LAMINATION

(not intended for definitive thermoformed sockets)

- Flatten distal end of Plaster Cast (9) to center Steel Guide (7). Attach with double-sided tape or two nails maintaining original pin alignment
- Apply inner PVA bag and tie it around *“neck”* of Steel Guide (7)
- Trim bag around string and place Silicone Seal (6) over *“neck”* of Steel Guide (7), onto PVA bag.
- Screw Fabrication Ring (4) (from locking component kit) onto Lock Dummy (5)
- Choose appropriate length of Release Dummy (8) (*“standard”* length when manufacturing small to medium sockets, *“long”* when manufacturing large sockets) and screw it FIRMLY into Lock Dummy (5) using 5mm hexagonal key
- Apply release agent to components to enable easy removal after

lamination (I-400300 Silicone Adhesive is recommended)

- Place the components onto Steel Guide (7) (make sure that o-ring on top of Lock Dummy is in place), Pyramid Cap (2) onto Pyramid (3) and secure them with Pyramid Bolt (1) using 6mm hexagonal key (make sure that o-ring on Bolt is in place),

IMPORTANT: Be sure to tighten so that resin can not leak in between components

- Apply clay into hexagon hole on Release Dummy (B) and Pyramid Bolt (C)
- Tie lay-up material into grooves on Fabrication Ring (4), Pyramid (3) and Pyramid Cap (2)

IMPORTANT: Tie at least two layers of fibers in groove on Pyramid Cap (2)

- Apply outer PVA bag and pour resin
- When curing has finished, remove Bolt (1) and Pyramid Cap (2), grind down to Release Dummy (8) and remove. DO NOT grind material away from distal end of socket since that reduces strength of connection between socket and pyramid
- Remove socket from plaster cast
- Remove Steel Guide (7) and Silicone Seal (6)
- Use Hex Driver (10) and a 8mm (5/16) socket wrench to release Lock Dummy (5)
- Insert appropriate locking component according to instructions enclosed

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

Limited one-year warranty
Rated up to high activity level (K3)

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- Innensechskantschlüssel 4mm and 5mm
- Steckschlüssel 8mm

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Icelock® 600 Series is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.