## ENGLISH

### CONTENT

L-792037 Alloy 4-Hole 774 (Aluminum tested to ISO10328, load level A60, rated to 100lbs/45kg for normal use) (**Fig. 3**).

L-793041 Screw M5x16 (4pc).

L-793112 Silicone Seal (**Fig. 5**) (only for lamination).

#### Use with L-780000 Fabrication Tooling (reusable).

#### When fabrication is complete, insert the following locking component.

L-721000 Icelock ratchet 721.

#### Tools required.

- 4mm hexagonal key.
- 8mm (5/16) socket wrench.
- 2mm rod.

#### LAMINATION

- Flatten distal end of Plaster Cast (8) to center Steel Guide (6). Attach with double-sided tape or two nails maintaining original pin alignment.
- Apply inner PVA bag and tie it around “neck” of Steel Guide (6).
- Trim bag around string and place Silicone Seal (5) over “neck” of Steel Guide (6), onto PVA bag.
- Screw 4-hole adapter (3) onto Lock Dummy (4).
- Choose appropriate length of Release Dummy (7) and screw it FIRMLY into Lock Dummy (4) using 4mm hexagonal key.
- Screw Pin Dummy Bolt (2) into shorter Pin Dummy (1) making sure that hexagon hole on Pin Dummy Bolt stays visible.
- Place the components onto Steel Guide (6) and secure them with Pin Dummy and Bolt (1;2) using 2mm rod (make sure that o-ring on top of Lock Dummy is in place).

**IMPORTANT:** Be sure to tighten so that resin can not leak in between components.

- Apply release agent to components (A) to enable easy removal after lamination (I-400300 Silicone Adhesive is recommended).
- Apply clay into hexagon hole on Release Dummy (B) and through hole on Pin Dummy (C).
- Tie lay-up material into groove on 4-hole adapter (3) and over components.
- Apply outer PVA bag and pour resin.
- When curing has finished, grind down to Release Dummy (7) and remove.
- Remove socket from plaster cast.
- Remove Bolt (2) in Pin Dummy (1) from inside of socket with 4mm hexagonal key.
- Remove Steel Guide (6) and Silicone Seal (5).
- Use Hex Driver (9) and 8mm (5/16) socket wrench to

release Lock Dummy (4).

- Remove Pin Dummy (1) with 4mm hexagonal key.
- Grind down to spacers on 4-hole adapter (3) and remove all four silicone sealings.
- Insert appropriate locking component according to instructions enclosed.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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