


LANYARD CORD WITH CONNECTING SCREW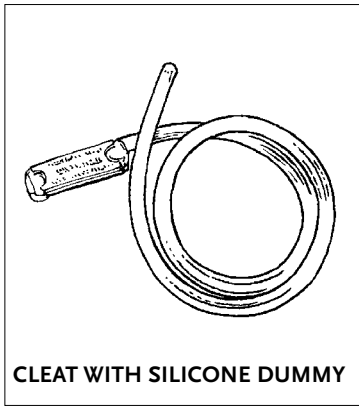








CLEAT WITH SILICONE DUMMY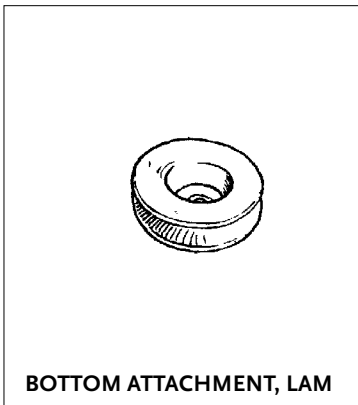

BOTTOM ATTACHMENT, LAM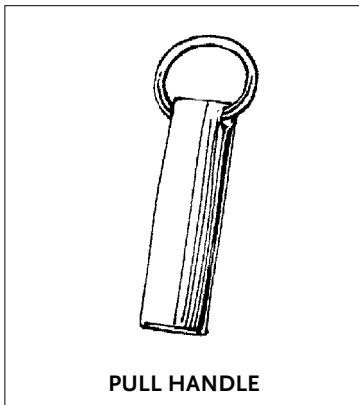

PULL HANDLE

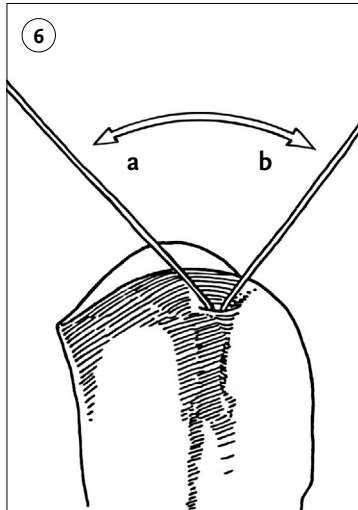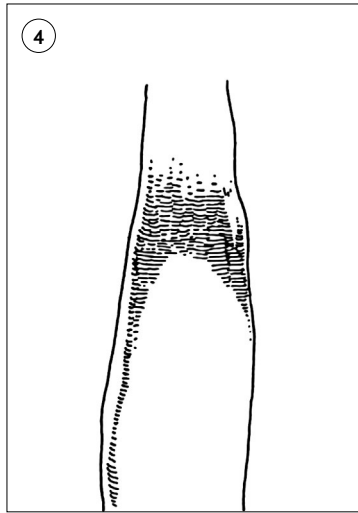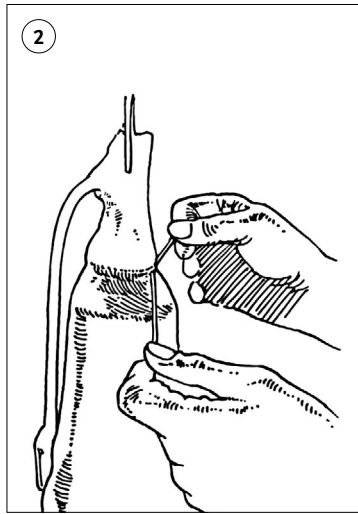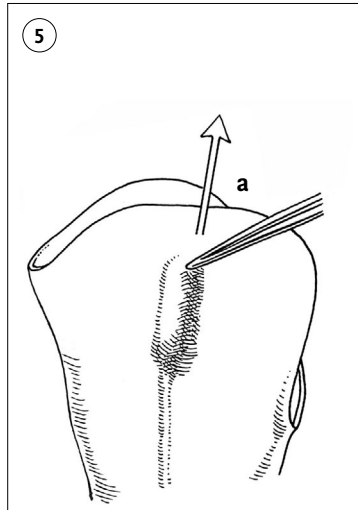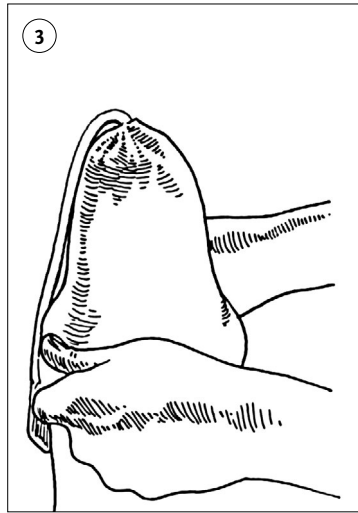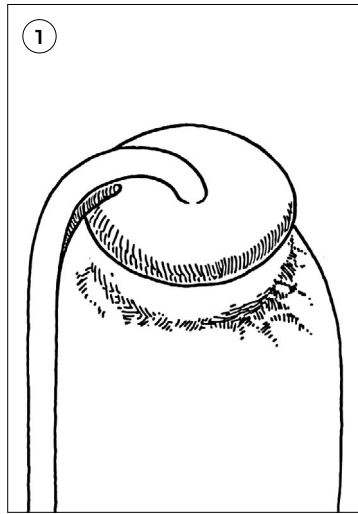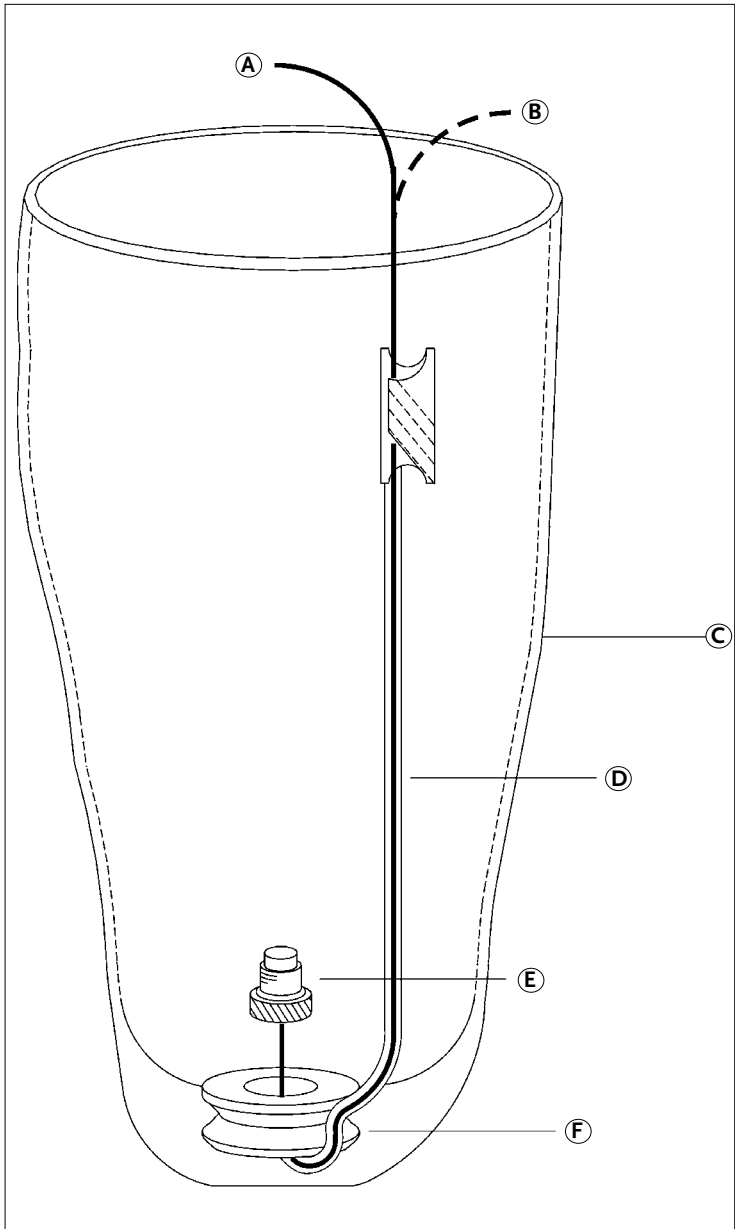ENGLISH

COMPONENTS:

- Bottom Attachment, lam
- Cleat with silicone dummy
- Pull Handle
- Lanyard Cord with connecting screw

- A. Released
- B. Locked
- C. Laminated socket
- D. Lanyard cord
- E. Connecting screw
- F. Bottom Attachment, lam

The ICELOCK lanyard 331-L is a component for the lanyard suspension of an ICEROSS prosthesis. It is intended to be used with laminated sockets where the cleat and channel for the lanyard are positioned in the laminate, offering a very neat cosmetic solution.

1. Pull the silicon dummy through the hole of the distal connector and make sure that it is positioned in the notch. To determine the length of the channel, position the cleat in its intended position on the model and shorten the dummy to the required length.
2. Pull the PVA bag over the model. Place the Bottom Attachment inside the bag. Tie the PVA bag in the groove on the Bottom Attachment and trim the excess. Make sure the knot in the groove is tied tightly enough to prevent the PVA bag from leaking air when the air is sucked out.
3. Apply half of the lay-up material and tie it in the groove of the Bottom Attachment. Position the Cleat in its

- intended position. Secure the position of the cleat with thread.
4. Apply the second half of the lay-up on top of the Cleat, tying off beyond the Bottom Attachment. Pull the outer PVA bag over the model and laminate.
5. When the socket has been removed, expose the silicone dummy at the proximal end of the cleat and pull the silicone out.
 - a - **Pull out**
6. Lead the Lanyard Cord through the channel from the inside of the socket with the help of compressed air. Tie a knot on the end of the lanyard to stop it inadvertently being pulled into the channel.
 - a - **Unlocked**
 - b - **Locked**

- A string can be attached to the trimmed end of the silicone dummy before laminating to facilitate leading the lanyard after lamination.
- The Pull Handle can be used for getting a good grip on the lanyard when the socked is donned.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Ossur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Ossur products are used with other recommended Ossur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©Copyright Ossur 2019