ENGLISH

CONTENT

P-20177	Step-less Distal Attachment
P-20170	Locking Pin
P-20161	Funnel
P-20141X	Coupling Plate
P-20141XX	Attachment Plate (3mm PP)

<i>To produce a socket with P-20191X you also need</i>	
P-20109	Fixture (reusable)
P-20151	Dummy (reusable)

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- Drill 3 holes into the Attachment Plate through the Coupling Plate. Secure the Coupling Plate to the Attachment Plate with enclosed 3 screws **(9)**.
- Trim push button to desired length and screw onto the Attachment **(10)**.
- Apply the Funnel (P-20161) to the Attachment from the inside of the socket **(11)**.

NOTE:

- Always check that the threaded adapter for the locking pin is tightened before the prosthesis is applied.
- The product has to be checked at least once a year by a prosthetist.
- The locking pin should be exchanged at least once a year.
- In case of replacements, only use original components.
- Maximum weight of the prosthesis: 10 kg.

Cleaning The Pin Step-less Distal Attachment:

Unscrew the Push Button. Blow with compressed air alternately between the Locking Pin hole and the Push Button hole. **NOTE!** Do not use lubricant or cleaning material. Only use air.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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