ENGLISH

CONTENT

P-20177	Step-less Distal Attachment
P-20170	Locking Pin
P-20161	Funnel
P-20141	Coupling Plate

To produce a socket with P-20191 you also need

P-20109	Fixture (reusable)
P-20151	Dummy (reusable)

TOOLS REQUIRED

3mm Drill Bit
Cross Head Screw Driver

SOCKET FABRICATION

- Flatten distal end of the plaster cast and attach the Fixture (P-20109) (1) with two small nails. Transition from cast to the Fixture should be smooth. Ensure the Fixture is mounted in alignment with the residual limb.
- Fabricate a soft liner of 3mm PeLite or similar material (2). Leave the distal bolt end open. Mount the bolt (3).
- Apply the Dummy (P-20151) on top of the Fixture. Make sure that the raising for the push button is orientated correctly (4).
- Insulate with A PVA-bag and seal at the top of the Dummy (5).
- Place chosen socket adapter on top of the Dummy, between the layers, and laminate as usual (6).
- Remove the cured socket from cast. Drill carefully to reveal the centre screw in the raising for the push button. Use a screwdriver or similar to push the plastic piece into the dummy. Remove the dummy from the socket. Grind a hole for the push button (7).
- Attach Coupling plate (P-20141) to the Attachment with enclosed 6 screws. Insert the Coupling Plate and Attachment into the socket (8).
- Drill 4x 3mm holes through the socket wall into the flange of the Coupling Plate. Secure the Coupling Plate to the socket with enclosed 4 screws (9).
- Trim the push button to desired length and screw onto the Attachment (10).
- Finally apply Funnel (P-20161) to the Attachment from inside of the socket (11).

Note:

- Do always check that the threaded adaptor for the locking pin is tightened before the prosthesis is applied.
- The product has to be checked at least once a year by a prosthetist.
- The locking pin should be exchanged at least once a year.
- In case of replacements, only use original components.
- Maximum weight of the prosthesis: 10 kg.

Cleaning The Pin Stepless Distal Attachment:

Unscrew the Push Button. Blow with compressed air alternately between the Locking Pin hole and the Push Button hole.

NOTE! Do not use lubricant or cleaning material. Only use air.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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