

PHILADELPHIA[®]
TRACHEOTOMY COLLAR



INDICATIONS C2 - C5

- Degenerative disorders
- Immobilisation after trauma/surgery
- Spinal Stenosis
- Spondylolisthesis

FEATURES AND BENEFITS

The Philadelphia Tracheotomy Collar offers immobilisation, comfort and enables healthcare professionals to quickly perform carotid pulse monitoring and emergency tracheotomies.

- Anterior and posterior rigid plastic reinforcement supports limited movement
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and aqua-therapy
- Easily customised for individual requirements
- Latex-free, non-toxic and hypoallergenic Plastazote® foam reduces skin irritation

MEASURING AND SIZING

Proper sizing is important for patient immobilisation and comfort. Always maintain the patient's head in neutral alignment.



HEIGHT

Measure from the tip of the chin to the top of the sternum in a straight line (Fig. A). If the patient's measurement falls between two consecutive sizes, apply the smaller size first.



NECK CIRCUMFERENCE

Measure around the patient's neck (Fig. B). The measurement corresponds to the size circumference of the Collar. If the patient's measurement falls between two consecutive sizes, apply the larger size first.

ORDERING INFORMATION

PHILADELPHIA® TRACHEOTOMY COLLAR

PART #	HEIGHT	SIZE	NECK CIRCUMFERENCE
PHP-T2XS	2.25"	XSmall	up to 10" / 25.4cm
PHP-T2S	2.25"	Small	10 - 13" / 25.4 - 33cm
PHP-T2M	2.25"	Medium	13 - 16" / 33 - 40.6cm
PHP-T2L	2.25"	Large	16 - 19" / 40.6 - 48.2cm
PHP-T2XL	2.25"	X Large	19"- up / 48.2cm - up
PHP-T3S	3.25"	Small	10 - 13" / 25.4 - 33cm
PHP-T3M	3.25"	Medium	13 - 16" / 33 - 40.6cm
PHP-T3L	3.25"	Large	16 - 19" / 40.6 - 48.2cm
PHP-T3XL	3.25"	X Large	19"- up / 48.2cm - up
PHP-T4S	4.25"	Small	10 - 13" / 25.4 - 33cm
PHP-T4M	4.25"	Medium	13 - 16" / 33 - 40.6cm
PHP-T4L	4.25"	Large	16 - 19" / 40.6 - 48.2cm
PHP-T4XL	4.25"	X Large	19"- up / 48.2cm - up
PHP-T5S	5.25"	Small	10 - 13" / 25.4 - 33cm
PHP-T5M	5.25"	Medium	13 - 16" / 33 - 40.6cm
PHP-T5L	5.25"	Large	16 - 19" / 40.6 - 48.2cm
PHP-T5XL	5.25"	X Large	19"- up / 48.2cm - up

PHILADELPHIA® COOLMAX® LINERS™

PART #	PHILADELPHIA COOLMAX LINER	SIZE*
PHP-LCXS	Philadelphia Coolmax Liner	X Small
PHP-LCS	Philadelphia Coolmax Liner	Small
PHP-LCM	Philadelphia Coolmax Liner	Medium
PHP-LCL	Philadelphia Coolmax Liner	Large
PHP-LCXL	Philadelphia Coolmax Liner	X Large

*Philadelphia Coolmax Liner sizes are identical to Philadelphia Collar sizes.

PHILADELPHIA COOLMAX LINER



Designed specifically for use with the Philadelphia Collars to enhance the patient's level of comfort during periods of extended wear and keep the Collar clean.



FOLLOW ÖSSUR ON



WWW.OSSUR.COM.AU

Össur Australia - OA & Injury Solutions
 TEL 1300 123 268 TEL +61 3 8761 6408
 FAX +61 3 9761 6067
 infomelbourne@ossur.com

