




Instructions for Use



---

PRO-FLEX XC TORSION  
W/UNITY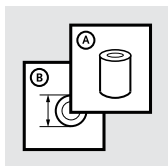

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	13
ES	Instrucciones para el uso	17
DA	Brugsanvisning	21
SV	Bruksanvisning	25
NL	Gebruiksaanwijzing	29
PT	Instruções de Utilização	33
JA	取扱説明書	37
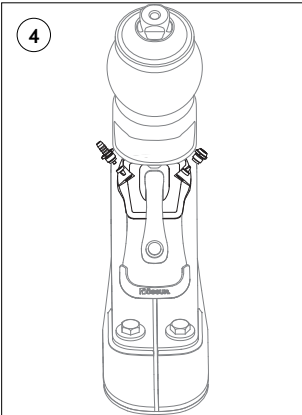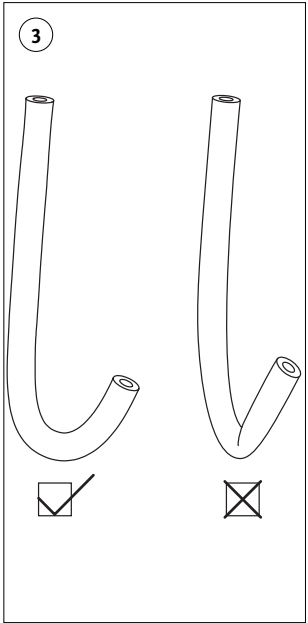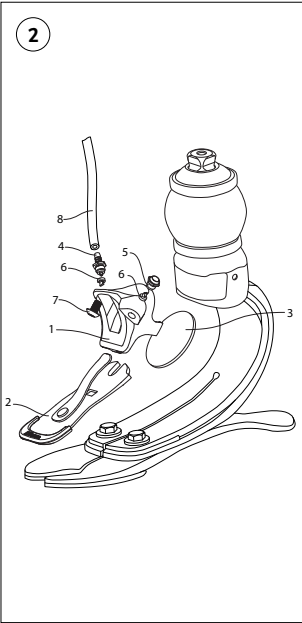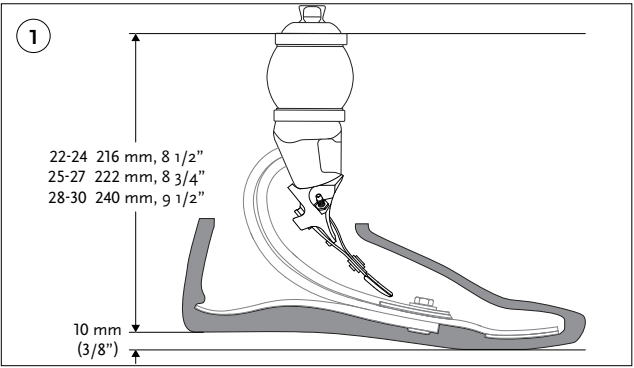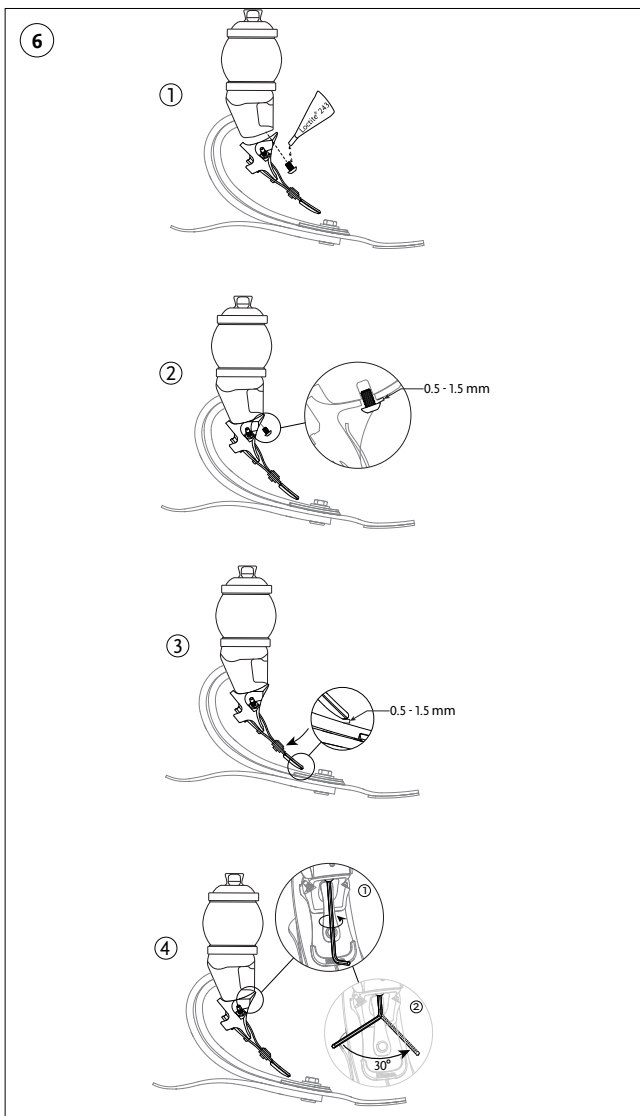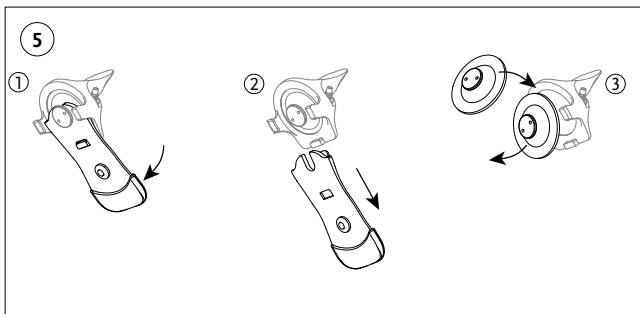
# ENGLISH

---

**The Unity® System** is referred to as the **device** in the following document. This document provides important information on the indications for use, and handling of the device. It is intended for use by a certified prosthetist and the user of the device. The device may only be fitted by a qualified practitioner authorized by Össur after completing the corresponding training.

## DEVICE COMPONENTS (Figure 2)

1. Membrane Housing
2. Pump Blade
3. Membrane
4. Inlet Port
5. Outlet Port
6. Duckbill
7. Mounting Screw
8. Tube

## PRO-FLEX XC TORSION FOOT

See IFU for Pro-Flex XC Torsion.

## SAFETY

**Warning:** A strong cautionary statement regarding possible dangers related to the intended use or foreseeable misuse of a device.

**Precaution:** A cautionary statement requiring action to be taken in advance of device use, or in the event of malfunction or change in device performance. Intended to ensure safe and effective use of device, and to protect the user against greater risk of harm and/or possible failure of the device, when used as intended.

## Precautions

- U.S. federal law requires that the device is sold by a prosthetist and by the order of a physician. The practitioner must provide each user with training for appropriate use and proper application of the device.
- The device is to be used exclusively for unilateral or bilateral major lower-limb amputations.
- Use a Unity® Socket Valve Kit to connect the device to the socket.
- Ensure the socket is a close fit to prevent blistering and/or wounds.
- Ensure the tube is not folded, bent, or closed off in any way (**Figure 3**).
- Ensure the tube is secured firmly away from any sharp objects or components.

**N.B.:** Practitioners must ensure their clients are aware of, and understand the following:

- The tube must be well secured at all times.
- Never operate the device if a change in its operation is observed. In such cases, the user should consult their practitioner immediately.

## DEVICE OVERVIEW

The device is designed to provide elevated vacuum for socket suspension. Expected attainable vacuum achieved during normal operation ranges from 15–18 inHg or higher.

Assisted suspension is obtained through a pump which draws air out from the socket in each step creating an elevated vacuum inside the socket interface.

The device is designed to function in any temperature and humidity that the user encounters during normal use.

The device is a single patient device.

**N.B.:** The device is designed to be used with Iceross Seal-In® liners, other applications are the sole responsibility of the provider.

## **INDICATIONS FOR USE**

Prosthetic interface with suspension properties for a major lower limb amputee; low to high impact use.

## **CONTRAINDICATIONS**

Not applicable.

## **STRUCTURAL SPECIFICATION**

- Net weight: Approximately 75g including tube.
- Frame construction: Carbon fiber and aluminum frame.

**Clearance and dimensions:** See IFU for Pro-Flex XC Torsion.

## **FITTING THE DEVICE**

**Warning:** Ensure the socket is fit securely before using the device; never use with a loose fitting or leaking socket.

**N.B.:** To connect the system to a socket, a Unity Socket Valve Kit is recommended. The use of other applications is the sole responsibility of the provider.

- Fit the user with a prosthetic foot containing the device
- Cut the tube to correct length, and connect to the device and socket (using the Unity Socket Valve kit).

Use the supplied tube fixture to secure the tube against the pylon. The tube should be located medially rather than laterally to avoid damage during ambulation. Once the device has been connected, ensure the pump is working properly, i.e. creating vacuum during gait and not leaking air.

## **DISASSEMBLING / ASSEMBLING THE DEVICE**

The device can be disassembled in case it needs to be cleaned or for troubleshooting purposes.

### **Precautions:**

- The device should only be disassembled / assembled by a practitioner.
- The inlet and outlet (**Figure 4**) ports on the membrane housing should not be disassembled unless absolutely necessary. This part of the device contains miniature duckbill valves that are easily lost and can cause the device to malfunction if not correctly reassembled.

### **Assembling the Device**

See Figure 6.

**N.B.:** Bumper size depends upon foot size and category.

## 1. AFTER DISASSEMBLY / ASSEMBLY

Check the following:

- Are the duckbill valves oriented in the correct direction (**Figure 4**)?
- Is the membrane fully inserted into the housing?
- Are the inlet and outlet ports tightened?
- Is the tube fully inserted on the inlet port? (Check both at the socket valve and the membrane housing)

## HOW TO CHANGE THE DEVICE MEMBRANE

1. Use the Flex-foot shoehorn (or equivalent) to remove the prosthetic foot from the foot cover.
2. Disconnect the tube from the device membrane housing.
3. Untighten the screw that fastens the membrane housing to the Pro-Flex XC Torsion pyramid.
4. Follow Figure 5.
5. Lubricate the membrane before sliding it into the housing with Iceross Clean&Simple Spray or equivalent.

## FINISHING / LINER OPTIONS

Össur recommends that the device is used with an Iceross Seal-In® liner.

**Precaution:** Make sure the Flex-Foot sock does not interfere with the device as it could compromise the function of the pump.

## THE TUBE

The tube is the most sensitive part of the device, and special attention should be paid to it for potential functional issues that might arise.

**Precaution:** Ensure that device tubing is not compressed or folded by the application of cosmetic finishing.

Ensure the tube is NOT:

- Folded (**Figure 3**).
- Pressed up against sharp objects.
- Punctured.
- Dangling during gait.
- Located laterally, as there is more likelihood of damage.

## TROUBLESHOOTING

Check the following:

- Is there leakage in the socket?
- Is the tube punctured?
- Is the membrane punctured or torn?
- Has the membrane been pulled out of the membrane housing?
- Is there dust or grit inside the device?
- Is the tube fully inserted on the inlet port? (Check both at the device socket valve or membrane housing)
- Are the inlet and outlet ports tightened?
- Is the device valve leaking?
- Is the sock inside the foot cover affecting device mechanical movement?

## MAINTENANCE

The following components of the device may need cleaning or replacing

during the lifetime of the device. This is to be considered normal wear, and is not covered by the Limited Product Warranty.

- Membrane
- Duckbill valves
- Tube

It is recommended to have the product inspected every 6 months by a practitioner.

- Clean the components with mild soap and rinse with water, isopropyl alcohol or Iceross Clean&Simple spray.
- DO NOT use solvents stronger than isopropyl alcohol.

Please refer to the current Össur printed catalog or online pages via: [www.ossur.com](http://www.ossur.com) for replacement parts available and appropriate item numbers.

## **WARRANTY**

The device has a 24 month limited warranty, excluding the membrane. The membrane has a 12 month limited warranty.

The following activities will void the warranty:

- Exposure to seawater or extremely corrosive environments.
- Use with copolymer gel liners.
- Use without liners or other means, which can lead to sweat or fluids other than Iceross Lubricant entering the device.
- Exposure to environments with extreme airborne particle contamination.
- Abuse and damage caused by abnormal impact.
- Dents and scratches on sealing surfaces.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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