ENGLISH

INDICATIONS FOR USE

Lower back conditions which may benefit from compression and gross motion restriction, such as:

- Lower back pain (Lumbago)
- Lumbosacral sprain/strain
- Degenerative joint disease

CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

WARNINGS

- The brace should not be used for extended periods of time, and removed during long periods of rest, e.g. sleep.
- This product is designed to be worn directly on the skin. Never wear the brace directly over an open wound. Inspect your back and the brace before and after each use. If any problems occur related to use, please contact your certified medical professional.

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- Contains natural rubber latex which may cause allergic reaction.
- LIST OF MATERIALS**
- Rubber, Polyester, Polyamide, Elastane, Cotton, Polyurethane, Steel, Styrene-Ethylene-Butylene-Styrene based Thermoplastic Elastomers (TPS-SEBS), Polypropylene, Polyacetal (POM).

CARE INSTRUCTIONS

Wash on a gentle cycle at max 30°C with mild detergent and air dry (do not tumble dry). For products with hook and loop please secure the hook and loop on the straps before washing to avoid damaging the knit. Please wash regularly to retain the compression of the knitted fabric.

FITTING INSTRUCTIONS

If between sizes on the sizing chart, always select the smaller size for optimal fit.

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www.ossur.com | ©Copyright Össur 2019 | IFU 1350_001 | Rev. 3 | NO7 06/2019