## ENGLISH

**INDICATIONS FOR USE**

For osteoarthritis and other knee conditions which may benefit from unloading and/or compression, such as:

- Medial or lateral knee pain
- Early or mild unicompartmental osteoarthritis of the knee
- Symptomatic, uni-compartmental meniscus degeneration

for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

- This product is designed to be worn directly on the skin. Never wear the brace directly over an open wound. Inspect your knee and the brace before and after each use. If any problems occur related to use, please contact your certified medical professional.
- Contains natural rubber latex which may cause allergic reaction.

**CONTRAINDICATION**

Must not be used by individuals for whom compression is contraindicated.

**WARNINGS**

The brace should not be used for extended periods of time, and removed during long periods of rest, e.g. sleep.

Patients with already compromised blood circulation in their extremities, like those with diabetes or peripheral vascular disease need to be extra careful, and are advised to consult with their doctor before using the brace. If you experience pain, swelling, sensation changes, or if your extremities show signs of insufficient blood flow (turning blue, white or cold) while using this brace, discontinue use immediately and consult with your medical professional.

- This product is designed and tested based on single patient usage, and is not recommended

##### LIST OF MATERIALS

Rubber, Polyester, Polyamide, Elastane, Cotton, Polyurethane, Ethylen-vinyl acetat (EVA), Polyacetat (POM).

##### CARE INSTRUCTIONS

Wash on a gentle cycle at max 30°C with mild detergent and air dry (do not tumble dry). For products with Velcro please secure the Velcro on the straps before washing to avoid damaging the knit. Please wash regularly to retain the compression of the knitted fabric.

##### FITTING INSTRUCTIONS

- If between sizes on the sizing chart, always select the smaller size for optimal fit.
- Please see available comfort sizes.
- To avoid damaging the knit, attach the Velcro straps appropriately.

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