## ENGLISH

### INDICATIONS FOR USE

Wrist conditions which may benefit from compression, such as:

- Overuse and pain conditions
- Wrist tendinitis
- Wrist sprain and strains
- Rheumatic Arthritis (RA) and other chronic pain conditions

wear the brace directly over an open wound. Inspect your wrist

- and the brace before and after each use. If any problems occur related to use, please contact your certified medical professional.
- Contains natural rubber latex which may cause allergic reaction.

### CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

### WARNINGS

- The brace should not be used for extended periods of time, and removed during long periods of rest, e.g. sleep.
- Patients with already compromised blood circulation in their extremities, like those with diabetes or peripheral vascular disease need to be extra careful, and are advised to consult with their doctor before using the brace. If you experience pain, swelling, sensation changes, or if your extremities show signs of insufficient blood flow (turning blue, white or cold) while using this brace, discontinue use immediately and consult with your medical professional.
- This product is designed to be worn directly on the skin. Never

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