## ENGLISH

### USES

Mild to moderate sprains, strains, post cast support and injury prevention.

### CAUTION

Patients should be instructed to inspect their ankle, foot, and product before and after each use. Always wear an absorbent sports sock.

### FITTING INSTRUCTIONS

- Step into the brace. Make certain the side stirrups are centered along the lower leg and ankle.
- Wrap the lower strap snugly around the brace.
- If necessary, adjust the heel width. Peel back the bottom layer of the hook and loop. Hold brace snugly against ankle and reapply hook and loop.
- Wrap top strap and readjust the lower strap as necessary to achieve a snug fit.
- Apply the shoe. Tighten laces.
- If necessary, inflate the aircells for additional compression and support. Inflate aircells by first peeling back the top portion of the bladder pads to locate the air valves. Insert the provided air tube through the air valve and inflate. **DO NOT OVERINFLATE.**

### WASHING INSTRUCTIONS

This support may be cleaned by wiping with a damp cloth and mild detergent. Do not use bleach or strong detergents. Do not wash in washing machine or place in dryer. Avoid direct contact for prolonged time in sun or direct heat.

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**EN – Caution:** This product has been designed and tested as a single use product, and is not intended for multiple use. In addition to increasing the risk of infection, used product components may contain small unnoticeable defects that may lead to non-optimal performance, and even mechanical failure. If any problems occur with the use of this product, immediately contact your medical professional.

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Airform® Inflatable Ankle Stirrup® is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.
**Össur Deutschland GmbH**