ENGLISH

INDICATION FOR USE:

- C-Spine precaution for trauma patients.
- Immobilization for pre C-Spine surgery.
- Other conditions requiring gross immobilization of the mid-cervical spine.

SIZE

One-piece

- Maintain neutral alignment. Measure from bottom of chin to top of shoulder at the neck (**Figure 1**).
- Position your lowest finger on the sizing line. Select the size closest to the height of your fingers (**Figure 2**).

Emt's choice

- Position your lowest finger on the sizing line. Select the size closest to the height of your fingers (**Figure 3**).
- Position the collar with the selected size. Press the lock into the collar until secured. Verify by attempting to readjust (**Figure 4**).

Patriot

- Position your lowest finger on the sizing line. Select the size closest to the height of your fingers (**Figure 5**).
- Position the collar with the selected size. Press the lock into the collar until secured. Verify by attempting to readjust (**Figure 6**).

APPLICATION

Supine position

- Maintain neutral alignment. Measure from bottom of chin to top of shoulder at the neck (**Figure 7**).

Seated position

- Position your lowest finger on the sizing line. Select the size closest to the height of your fingers (**Figure 8**).

Secure collar

- Position your lowest finger on the sizing line. Select the size closest to the height of your fingers (**Figure 9**).

PRECAUTIONS

- **Do not** rely solely on extrication collar for sufficient immobilization.
- **Do not** use improperly sized collars.
- **Always** follow local ems authority directions.

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© Copyright Össur 2018